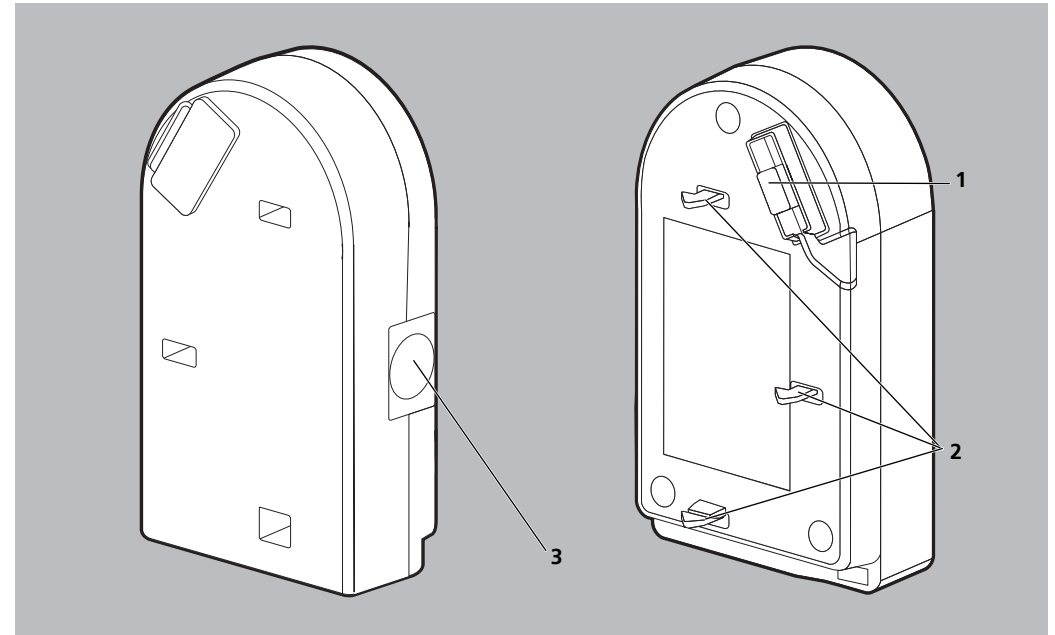
WM 67955e 01/2024 DE, EN, FR

Modul WM110MW

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Air pressure range	700 hPa - 1060 hPa
Type of protection against electric shock	Protection class III
IP classification to ISO 60529: Degree of protection against harmful ingress of solids and water	IP21
Electrical rating	12V DC
Power capacity* • in flight mode • when transmitting data	0.3 W up to 1.2 W
Electrical safety	See declaration of conformity of the manufacturer.
Electromagnetic compatibility EMC	Applied standards and wireless standards can be found in the modem manufacturer's declaration of conformity.
Wireless standards	Test parameters and limit values can be obtained from the modem manufacturer on request.

* Data apply only in combination with the licensed therapy device.

The right to make design modifications is reserved.

1 Introduction

1.1 Intended use

The modem WM110MW is for making a mobile wireless connection between therapy devices WM100TD, WM110TD or WM120TD and the prisma CLOUD platform. The module is intended for indoor use only.

1.2 Description of function

If the modem is connected to a compatible therapy device, this supplies it with power and controls it. The therapy device uses the modem to transmit data to the prisma CLOUD platform automatically at specific intervals. Progress and any data transmission errors are shown in the therapy device display. The display also indicates the signal strength of the connection to the mobile wireless network in the icon bar in the form of up to 5 green bars.

An automatic data transmission error may occur if it is not possible to connect to the wireless network, for example. If an error occurs during data transmission, a fresh attempt to transmit data will be started after a short time. Data are never transmitted during therapy and the modem is in so-called flight mode (no wireless signals are emitted).

During data transmission the therapy device can poll new settings from prisma CLOUD. The specialist dealer can deactivate this function on the device. As soon as new settings are received, a display informs the user of this.

2 Safety

2.1 Safety information

Risk of injury from operating the device outside the specified ambient conditions!

Using the device outside the specified ambient conditions may lead to tolerances being exceeded and to device failure, harming the patient.

- ⇒ Only operate device within the specified ambient conditions (see section entitled "Technical data").
- ⇒ Only operate the device in enclosed spaces.
- ⇒ Do not operate the device in vehicles or aircraft.
- ⇒ Do not operate the device in environments at risk of explosion, such as gas stations.
- ⇒ Maintain a minimum distance of 20 cm between the modem and parts of the body.

2.2 General information

- Have repairs and maintenance carried out only by the manufacturer or by specialist staff expressly authorized by the manufacturer.

- The owner/operator is responsible for ensuring the compatibility of the therapy device and of all the components or accessories connected to the patient before use. Have modifications to the device carried out only by the manufacturer or by specialist staff expressly authorized by the manufacturer.
- The device is subject to special precautions with regard to EMC (electromagnetic compatibility). It is essential to maintain the specified safety distance between the device and other devices which emit HF radiation (e.g. cell phones) in order to prevent malfunctions. Safety distances can be found in the Instructions for Use for the therapy device in Section "Safety distances".
- To prevent infection or bacterial contamination, follow the hygiene treatment section (see section entitled "Hygiene treatment").
- The Instructions for Use for the therapy device, the components and the accessories should also be followed.

3 Product description

3.1 Overview

A diagram of the individual parts can be found on the title page.

1. System interface connector for connection to the therapy device or to other modules.
2. Latches to secure the module to the therapy device or to other modules.
3. Release catch to connect or remove the module.

3.2 Symbols in the display of the therapy device

Symbol	Description
	Wireless icon flashing white: The connected modem has been detected.
	Wireless icon flashing green: The connected modem is making a connection.
	The connected modem is in flight mode. No wireless signals transmitted.
	The connected modem is transmitting data. The number of green bars indicates signal strength.

3.3 Markings and symbols

Symbol	Description
TYPE	Type designation
REF	Article number
IMEI	International Mobile Equipment Identity Number
ICCID	Integrated Circuit Card Identity Number
	Degree of protection against electric shock: Protection class III device
	Manufacturer
IP21	Degree of protection against solid foreign bodies. Device is protected against drips.
	Follow Instructions for Use.
	Wireless symbol
CE	CE marking
	Do not dispose of the device in domestic waste
	Direct voltage
GTIN	Global Trade Item Number
	Permitted storage temperature: -25 °C to 70 °C
	Permitted storage humidity: 15 % to 85 % relative humidity
IC	IC marking (Industry Canada of Canada)

4 Installation

Condition: The therapy device is on standby.

- Place the therapy device on a level surface (e.g. a table).
- Open the cover for the system interface on the right-hand side of the therapy device (see Fig. 1).
- Keep the release catch on the module depressed.
- Push the latches of the module into the latching bores of the therapy device or of a module already connected.

The modem must always be the last module to be connected, as it is itself unable to support connection of a further module!

- Push the module against the device until it engages.

After about 3 seconds, the  symbol appears in the therapy device display and the modem is operational.

4.1 Remove the modem

It may be necessary to disconnect the modem from the therapy device to clean it or to return it to your specialist dealer. Otherwise, the modem can stay permanently attached to the therapy device.

- Press the release catch on the module (see Fig. 2).
- Take the module off the side of the therapy device.
- Seal off the system interface on the therapy device again using the cover.

5 Hygiene treatment

5.1 General information

- Wear appropriate safety gear for the disinfecting process.
- Refer to the Instructions for Use for the disinfectant used.

5.2 Intervals

Clean the modem weekly. Disinfect the modem if required (e.g. following infectious diseases or severe contamination) or on change of patient. Sterilizing is not permitted.

5.3 Hygiene treatment for modem

NOTICE

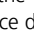
Risk of material damage as a result of ingress of liquids!

The modem may be damaged by the ingress of liquids.
⇒ Do not immerse the modem in liquids.

- Disconnect the modem from the therapy device (see the section entitled "4.1 Remove the modem").
- Wipe the modem over with a soft cloth.
- If necessary: Remove more stubborn particles of dirt with a soft cloth moistened with water or disinfectant (recommended products: terralin® protect or perform advanced Alcohol EP).

6 Function check

Carry out a function check after any hygiene treatment or repair, but at least every 6 months.

- Set up the therapy device properly in accordance with the Instructions for Use.
- Check the modem housing for cracks, damage and severe soiling.
- Connect the modem to the therapy device (see the section entitled "4. Installation").
- Check whether the wireless symbol appears in the therapy device display ().
- If the modem is not working properly or is damaged: Do not use device and contact your specialist dealer.

7 Faults

If you are unable to remedy faults with the aid of the table, or in the event of unexpected operation or an incident, contact your authorized specialist dealer to have the device repaired. To avoid exacerbating the damage, do not continue operating the device.

Fault	Cause	Remedy
No wireless symbol appears in the therapy device display, not even over ten seconds after connection of the modem	Modem not started correctly.	Disconnect the modem from the therapy device, reconnect after 5 seconds and wait about 15 seconds.
	Therapy device firmware does not support modem.	Contact your specialist dealer.
	Modem faulty.	Contact your specialist dealer.
Therapy device displays "No PIC set"	Therapy device is not configured for data transmission.	Contact your specialist dealer.
When the therapy start button is pressed, the therapy device displays "Please wait"	Temporarily unable to interrupt process.	Wait until "Please wait" disappears. This can take up to 90 seconds. To start therapy straight away: Disconnect modem (see "4.1 Remove modem").
Therapy device displays "Err 628"	Modem not supported.	Contact your specialist dealer.
Therapy device displays "Err 622"	Mobile wireless not configured.	Contact your specialist dealer.

Fault	Cause	Remedy
Therapy device displays  in the icon bar	No mobile wireless network available.	Try again later.
		Error occurs relatively frequently: Select a location with better reception.
		No remedy possible: Contact your specialist dealer.
Therapy device displays "Err 629"	Mobile wireless network not providing a data service.	Try again later. No remedy: Contact your specialist dealer.
Therapy device displays "Err 624"	No reliable link to remote data transmission station.	Try again later. No remedy: Contact your specialist dealer.
Therapy device displays "Err 627"	Configuration of remote data transmission station faulty.	Contact your specialist dealer.
Therapy device displays "Err 612"	Date of therapy device deviating too far from server date.	Contact your specialist dealer.
Therapy device displays "Err 616"	Device configuration via modem failed.	Contact your specialist dealer.

8 Servicing

The modem is designed for a service life of 6 years. If used in accordance with its intended use, the modem requires no maintenance according to these Instructions for Use.

If the device is used beyond this period, it needs checking by an authorized specialist dealer.

9 Scope of supply

Part	Article number
2G modem	WM 31240
4G modem (Europe)	LMT 31831
4G modem (Canada)	LMT 31833

10 Technical specifications

Temperature range Operation Storage	+5 °C to +40 °C -25 °C to +70 °C
Permitted humidity for operation and storage	15 % to 85 % relative humidity, no condensation

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