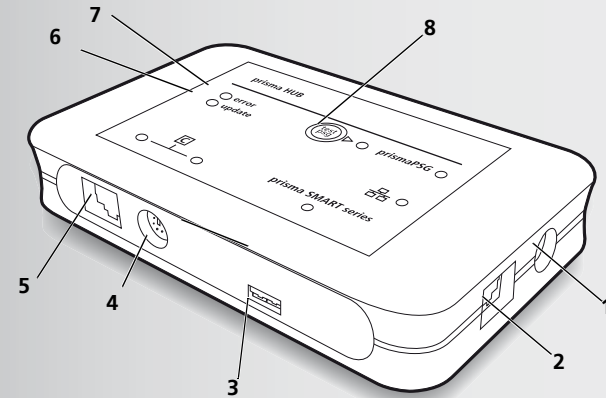
**DE** Gebrauchsanweisung für Module des Typs WM090MC **EN** Instructions for Use for WM090MC modules **FR** Mode d'emploi pour modules du type WM090MC

WM 68240b 02/2021 DE, EN, FR

**Löwenstein Medical  
Technology GmbH + Co. KG**  
Kronsaalsweg 40  
22525 Hamburg, Germany  
T: +49 40 54702-0  
F: +49 40 54702-461  
[www.loewensteinmedical.de](http://www.loewensteinmedical.de)

## prisma HUB

Kommunikationsmodul für Therapiegeräte des Typs WM090TD / WM100TD / WM110TD / WM120TD  
Communication module for therapy devices of type WM090TD / WM100TD / WM110TD / WM120TD  
Module de communication pour les appareils de thérapie du type WM090TD / WM100TD / WM110TD / WM120TD

1

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## 11 Accessories and replacement parts

A current list of accessories and replacement parts can be ordered on the internet site of the manufacturer or through your authorized specialist dealer.

## 12 Technical data

|  |  |
|--|--|
| Product class to MDR (EU) 2017/745   | I  |
| Dimensions W x H x D in cm   | 15 x 3 x 9.2   |
| Weight   | 0.2 kg   |
| Temperature range operation transport and storage                          | +5 °C to +40 °C<br>-25 °C to +70 °C                                |
| Permitted humidity for operation, transport and storage                    | 15 % to 93 %, no condensation                                      |
| Air pressure range   | 700 hPa - 1060 hPa, corresponds to an altitude of 3000 m above MSL |
| Classification to EN 60601-1-11: type of protection against electric shock | Protection class II  |
| Classification to DIN EN 60601-1: operating mode                           | Continuous duty  |
| Electrical rating  | 12 V/24 V  |
| Power capacity   | 2 VA   |
| Insulation of Ethernet port to PC  | 4 kV   |
| Insulation of Ethernet port to prismaCONNECT module                        | 1.5 kV   |

The right to make design modifications is reserved.

## 13 Warranty

Löwenstein Medical Technology gives the customer a limited manufacturer warranty on a new genuine Löwenstein Medical Technology product and on any replacement part fitted by Löwenstein Medical Technology in accordance with the warranty conditions applicable to the product in question and in accordance with the warranty periods from date of purchase listed below. The warranty conditions are available on the website of the manufacturer. We will also send you the warranty conditions on request. In the event of a claim under warranty, contact your specialist dealer.

| Product  | Warranty periods |
|--|------------------|
| Devices including accessories (except masks)   | 2 years          |
| Masks including accessories, rechargeable batteries, batteries (unless quoted differently in the technical documentation), sensors, patient circuits | 6 months         |
| Disposable products  | None             |

## 14 Declaration of conformity

Löwenstein Medical Technology GmbH + Co. KG, Kronsaaßweg 40, 22525 Hamburg, Germany, the manufacturer of the devices described in these Instructions for Use, hereby declares that the product complies with the respective regulations of Medical Devices Directive 93/42/EEC. The unabridged text of the Declaration of Conformity can be found on the manufacturer's website.

## 1 Introduction

### 1.1 Intended use

The WM090MC module is used to create a link between approved devices and a PC via Ethernet. It is also used to create a link between approved devices and the WM100MP module. The WM090MC module is designed solely for use indoors. The WM090MC module is not designed for use in domestic settings.

## 2 Safety

### 2.1 Safety information

#### Risk of injury from device or component malfunctions!

A damaged device or damaged components may injure the patient, the user and people in the vicinity.

⇒ Only operate the device and its components if they are externally undamaged.

⇒ Only operate the device and its components if a function check has been completed successfully.

#### Risk of injury from operating the device outside the specified ambient conditions!

Using the device outside the specified ambient conditions may lead to tolerances being exceeded and to device failure and injury to the patient.

⇒ Only operate device within the specified ambient conditions (see section entitled "Technical data").

⇒ Only operate the device in closed rooms.

### 2.2 General information

- Connect only the devices and modules permitted in accordance with these Instructions for Use. Devices connected to the LAN port of the module must meet their respective product standard. Non-medical equipment should be positioned out of the patient's vicinity.
- The owner/operator is responsible for ensuring the compatibility of the therapy device and of all the components or accessories connected to the patient before use. Have modifications to the device carried out only by the manufacturer, Weinmann, or by specialist staff expressly authorized by the manufacturer.
- If the WM090MC module is integrated in an IT network which includes other devices, the owner/operator must check whether this causes risks to patients, operators or third parties (IEC 80001-1).

- The use of third-party articles may lead to incompatibility with the device. In such cases, please be aware that any claim under warranty and liability will be void if neither the accessories nor the genuine replacement parts recommended in the Instructions for Use are used.
- To prevent infection or bacterial contamination, follow the section about hygiene treatment (see section entitled "Hygiene treatment").
- In the EU: As a user and/or patient, you must report any severe adverse events occurring in connection with the product to the manufacturer and to the responsible authorities.

## 3 Product description

### 3.1 Overview

A diagram of the individual parts can be found on the title page.

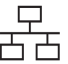
Key

1. Connection for prismaPSG (WM100MP)
2. LAN port for connecting to a PC.
3. Port for prisma SMART series (WM090TD)
4. PSG port to the prismaCONNECT module (WM100MC).
5. LAN port to the prismaCONNECT module (WM100MC).
6. Update LED - indicates an update in progress.
7. Error LED - indicates a fault.
8. PSG key - outputs test signals to the prismaPSG.

### 3.2 Ferrite

Ferrite (see **Figure 1** and **Figure 2**) is a non-conductive material which absorbs electromagnetic waves passing through it. It serves as an interference filter which eliminates electromagnetic waves before they can be emitted via the cable.

## 4 Marks and symbols

| Symbol  | Description  |
|---|--|
|    | Network connection: LAN interface to the PC  |
|    | Interface to the prismaCONNECT module  |
|    | Do not dispose of the product in domestic waste  |
|    | Degree of protection against electric shock: Protection class II product                         |
|    | Protect from moisture  |
|    | Permitted temperature range for transport and storage  |
|    | Permitted humidity range for transport and storage   |
|    | Manufacturer and date of manufacture   |
|    | Follow the instructions for use  |
|  | Serial number  |
|  | Order number   |
|  | Indicates the product is a medical device  |
|  | Unique device identifier   |
|  | CE symbol (confirms that the product conforms to the applicable European directives/regulations) |

## 5 Operation

### 5.1 Connecting the module

1. Make plug connections in line with **Figure 1** or **Figure 2**.

 You can use the module with the following devices:

- prisma SMART series (WM090TD)
- prismaLINE series (WM100TD) (in conjunction with the prismaCONNECT module)
- prisma VENT series (WM110TD / WM120TD) (in conjunction with the prismaCONNECT module)

Do not connect several devices simultaneously.

The relevant LED comes on.

2. Attach ferrites to the cables in line with **Figure 1** or **Figure 2**, attaching the ferrites as close as possible to the prisma HUB module.

### 5.2 Switching on the module

1. To switch on the module: connect the module to a therapy device.  
The LEDs which have a plug connection come on. The LEDs go out one minute after the last change.

2. To switch on the LEDs: press PSG key **8**..

### 5.3 Outputting a PSG test signal

1. Press PSG key **8**..  
The LED comes on. A synthetic signal is output to the PSG system for 2 minutes.

### 5.4 Configuring the module

The DHCP preset is activated at the LAN port for connection to a PC **2**.. A dynamic IP address is assigned to the prisma HUB module. If no DHCP server is available, the IP address is assigned by zeroconf/auto-IP in the range 169.254.1.0 to 169.254.254.255.

You can assign a static IP address to the module with the aid of the prisma HUBconfig PC software.

### 5.5 Integrating the module in an IT network

The module can be integrated in an IT network to create a link between the module and a PC. The network must have the following properties:

- UDP and TCP ports 51337 and 51338 must be open and may not be blocked by a firewall.
- The UDP protocol is required to identify devices in the network automatically, data exchange subsequently takes place using the TCP protocol.

If the IT network fails, you can continue operating the therapy device via the display and transmitting data via SD card. This prevents failure of the IT network giving rise to hazardous situations.

## 6 Hygiene treatment

### 6.1 General information

- Wear appropriate safety gear for the disinfecting process.
- Refer to the Instructions for Use for the disinfectant used.

### 6.2 Intervals

Clean the module weekly. If required (e.g. following infectious diseases or severe contamination), disinfect the module. Sterilizing is not permitted.

### 6.3 Subjecting module to a hygiene treatment

#### NOTICE

#### Risk of material damage as a result of ingress of liquids!

The module may be damaged by the ingress of liquids.

⇒ Do not immerse the module in liquids.

1. Detach all connections from the module.
2. Wipe the module over with a soft cloth.
3. If necessary: remove more stubborn particles of dirt with a soft cloth moistened with water or disinfectant (recommended products: terralin® protect or perform advanced Alcohol EP).

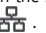
## 7 Function check

Carry out a function check after every hygiene treatment and repair, but at least every 6 months.

1. Remove all cables from the module one after another.  
The relevant LED goes off.
2. Re-connect all the cables to the module one after another.  
The relevant LED comes on.

3. Press **PSG** key **8**..  
The LED comes on. A synthetic signal is output to the PSG system for 2 minutes.
4. If the module is not working properly or is damaged: contact your specialist dealer.

## 8 Troubleshooting

| Fault   | Cause  | Remedy   |
|---|--|--|
| Error LED <b>7</b> is on                            | Technical fault  | Have module repaired.  |
| No LAN connection possible                          | Cable plugged in incorrectly                                   | Check whether the LAN cable is in the socket marked  .  |
| No connection to the prismaCONNECT module           | Cable plugged in incorrectly or not at all.                    | Check whether the PSG cable and the LAN cable are in the sockets marked  and the LEDs are on. |
| No connection to the prismaPSG module               | Cable plugged in incorrectly                                   | Check whether the PSG cable is in the socket marked <b>prismaPSG</b> and the LED is on.  |
| No therapy device is found in the prismaTS software | IP address of the prismaHUB configured incorrectly             | Check IP settings with the aid of the prismaTS software.   |
|   | The prismaCONNECT module was configured to a static IP address | Configure prismaCONNECT module to DHCP   |
|   | Firewall blocking data transmission                            | Enable UDP and TCP ports 51337 and 51338.  |

## 9 Servicing

The module is designed for a service life of 6 years. If used in accordance with its intended use, the module requires no maintenance according to these Instructions for Use.

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