DE Gebrauchsanweisung für Module des Typs WM100MC EN Instructions for  
Use for WM100MC Modules FR Mode d'emploi pour modules tu type WM100MC

## prismaCONNECT

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# Contents

<b>1</b>	<b>Introduction</b>	<b>30</b>
1.1	Intended use .....	30
<b>2</b>	<b>Safety</b>	<b>31</b>
2.1	Safety information .....	31
2.2	General information .....	32
2.3	Warnings in this document .....	33
<b>3</b>	<b>Product description</b>	<b>34</b>
3.1	Overview of prismaCONNECT module .....	34
3.2	Markings and symbols .....	35
<b>4</b>	<b>Preparation and operation</b>	<b>37</b>
4.1	Connecting the prismaCONNECT module .....	37
4.2	Removing the prismaCONNECT module .....	39
4.3	Configuring the prismaCONNECT module .....	40
4.4	Integrating the prismaCONNECT module into an IT network .....	41
<b>5</b>	<b>Hygiene treatment</b>	<b>42</b>
5.1	General information .....	42
5.2	Intervals .....	42
5.3	Subjecting module to a hygiene treatment .....	42
<b>6</b>	<b>Function check</b>	<b>44</b>
6.1	Intervals .....	44
6.2	Performing a function check .....	44
<b>7</b>	<b>Faults</b>	<b>45</b>
<b>8</b>	<b>Servicing</b>	<b>46</b>
<b>9</b>	<b>Transport, storage and disposal</b>	<b>47</b>
9.1	Transport and storage .....	47
9.2	Disposal .....	47
<b>10</b>	<b>Annex</b>	<b>48</b>
10.1	Technical specifications .....	48

10.2	Scope of delivery .....	49
10.3	Accessories .....	49
10.4	Spare parts .....	49
10.5	Warranty .....	49
10.6	Declaration of Conformity .....	50

# 1 Introduction

## 1.1 Intended use

The WM100MC module is used to create a connection between the approved therapy devices and a PC via Ethernet. It is also used to create a connection between the approved therapy devices and the WM100MP module. Module WM100MC is designed solely for use indoors. The WM100MC module is not intended for use in a domestic environment.

## 2 Safety

Read these instructions for use carefully. They are a constituent part of the devices described and must be available at all times.

Use the device exclusively for the intended purpose described ([see "1.1 Intended use", page 32](#)).

For your own safety and the safety of your patients and in accordance with the requirements of (EU) Regulation 2017/745, follow the safety information below.

### 2.1 Safety information

- Warning*     **Risk of injury from device or component malfunctions!**  
 A damaged device or damaged components may injure the patient, the user, and bystanders.
- ⇒ Only operate the device and its components if they are externally undamaged.
  - ⇒ Only operate the device and its components if a function check has been completed successfully.
- Risk of injury from operating the device outside the specified ambient conditions!**  
 Using the device outside the specified ambient conditions may lead to tolerances being exceeded and to device failure, injuring the patient.
- ⇒ Only operate device within the specified ambient conditions ([see "10.1 Technical specifications", page 50](#)).
  - ⇒ Only operate the device indoors.
- Notice*     **Material damage due to dirt in the device!**  
 Dirt which penetrates the device during transport may damage it.
- ⇒ Only transport the device with the system interface cover closed.
  - ⇒ Transport the device in the associated packaging.

## 2.2 General information

- If third-party articles are used, malfunctions and restricted fitness for use may result. Biocompatibility requirements may also fail to be met. In such cases, please be aware that any claim under warranty and liability will be void if neither the accessories nor the original spare parts recommended in the instructions for use are used.
- Have measures such as repairs, servicing, and maintenance work carried out by the manufacturer or by specialists expressly so authorized by the manufacturer.
- Connect only the devices and modules approved in accordance with these instructions for use. Devices connected to the LAN port of the prismaCONNECT module must meet their respective product standard. Non-medical equipment should be positioned out of the patient's vicinity.
- The operator is responsible for ensuring the compatibility of the therapy device and of all the components or accessories connected to the patient before use.  
Have modifications to the device carried out only by the manufacturer or by specialist staff expressly authorized by the manufacturer.
- When integrating the WM100MC module into an IT network which includes other devices, the operator must check whether this creates risks for patients, operators or third parties (IEC 80001-1).
- To prevent infection or bacterial contamination, follow the hygiene treatment section ([see "5 Hygiene treatment", page 44](#)).
- You should also follow the instructions for use of the therapy device, the components, and the accessories.

- In the EU: As a user and/or patient, you must report any serious incidents occurring in conjunction with the product to the manufacturer and to the responsible authority.

## 2.3 Warnings in this document

Warnings indicate information relevant to safety.

Within procedures, you will find warnings in front of a step which presents a risk to persons or objects.

Warnings consist of

- the warning symbol (pictogram),
- a signal word to indicate the level of hazard,
- information about the hazard and
- instructions on how to avoid the hazard.

There are three hazard levels for warnings depending on the degree of risk:

 **DANGER**

### **Danger!**

Indicates an unusually significant hazardous situation. If you ignore this instruction, severe irreversible injuries or death will result.

 **WARNING**

### **Warning!**

Indicates an unusually significant hazardous situation. If you ignore this instruction, severe irreversible or fatal injuries may result.

 **CAUTION**

### **Caution!**

Indicates a hazardous situation. If you ignore this instruction, mild or moderate injuries may result.

**NOTICE**

### **Notice!**


Indicates a harmful situation. If you ignore this instruction, material damage may result.

Indicates useful information within procedures.

## 3 Product description

### 3.1 Overview of prismaCONNECT module

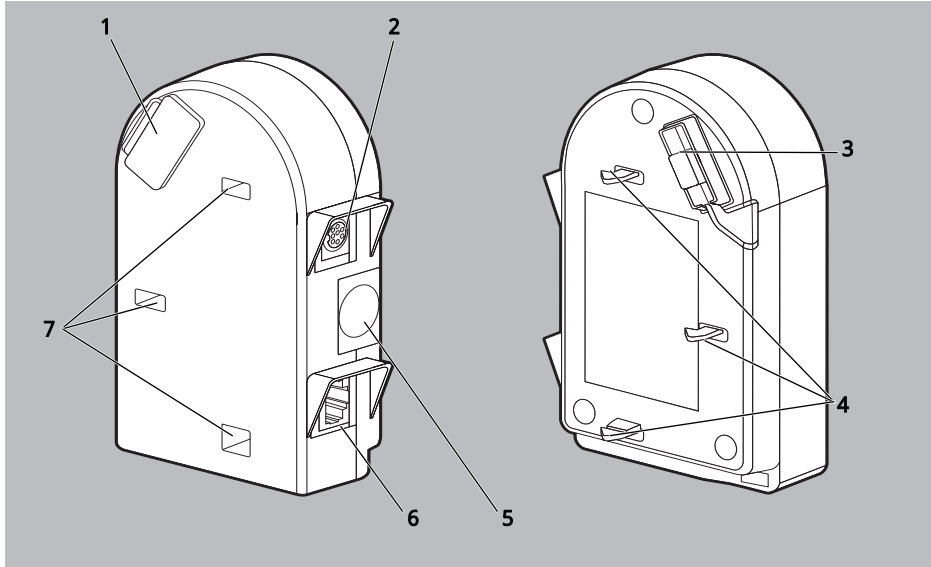3-1 prismaCONNECT module


No.	Designation	Description
1	System interface cover	Covers the system interface when no other module is connected.
2	Module interface	For connecting to the prismaPSG module or prisma HUB.
3	System interface	For connecting to the therapy device or to other modules.
4	Locking hook	Attaches the module to the therapy device or to other modules.
5	Module release catch	Allows the module to be connected to and removed from the therapy device.
6	LAN port	For connecting to a PC

No.	Designation	Description
7	Locking bores	Accommodate the locking hooks of other modules.

## 3.2 Markings and symbols

The following symbols may be applied to the device, the device ID plate, accessories or packaging.

Symbol	Description
	Serial number
	Indicates the product is a medical device
	Unique device identifier
	Order number
	Manufacturer and date of manufacture
<b>IP21</b>	Degree of protection against contact with a finger. Product is protected against vertically-falling drips of water
	Do not dispose of the product in domestic waste
	Degree of protection against electric shock: Protection class II product
	Follow the instructions for use
	CE symbol (confirms that the product conforms to the applicable European directives/regulations)
	Permitted humidity range for transport and storage

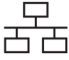Symbol	Description
	Permitted temperature range for transport and storage
	Identification of the prismaCONNECT module
	Network connection

## 4 Preparation and operation

### 4.1 Connecting the prismaCONNECT module


You can use the module with the following therapy devices and modules:

- prismaLINE series (WM100TD)
  - prisma VENT series (WM100TD/WM120TD)
  - prismaPSG (WM100MP) - connection and use are explained in the instructions for use of the prismaPSG module.
  - prisma HUB (WM090MC) - connection and use are explained in the instructions for use of the prisma HUB module.
1. Place the therapy device on a level surface (e.g., a table).
  2. Switch on the therapy device (see instructions for use of the therapy device).

You can also connect the module when the therapy device is switched off. Then switch on the therapy device.

3. Open the system interface cover on the right-hand side of the therapy device.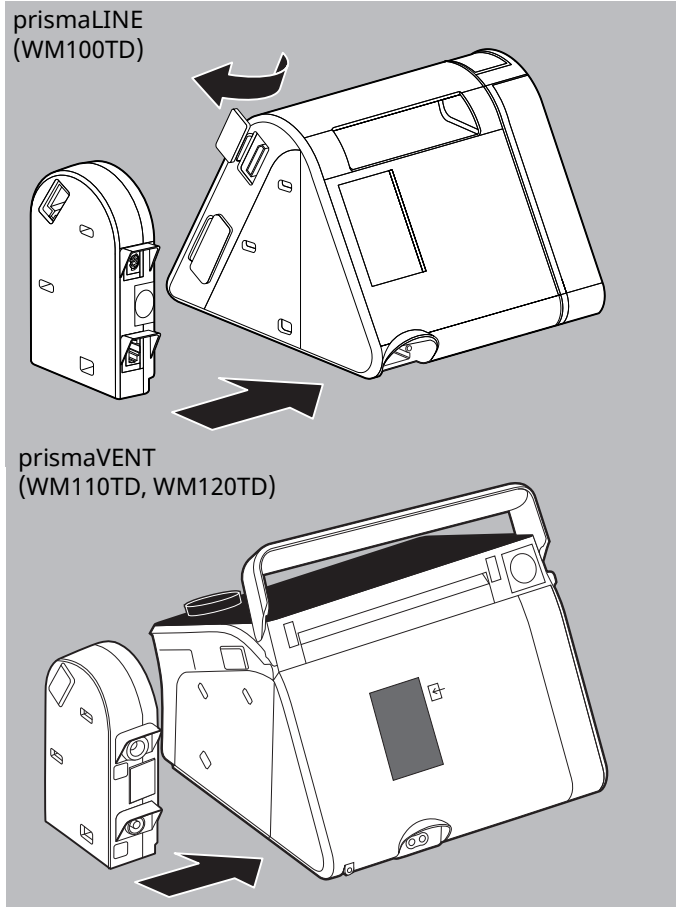

4. Keep the release catch on the module depressed.
5. Push the locking hooks of the module into the locking bores of the therapy device.
6. Push the module against the device until it clicks into place.

The **C** symbol appears in the display of the therapy device.


7. Connect the Ethernet cable to the LAN port and to a PC or an IT network.

The  symbol appears in the display of the therapy device.

*Result* prismaCONNECT has been connected.

## 4.2 Removing the prismaCONNECT module

1. Press the release catch on the module.

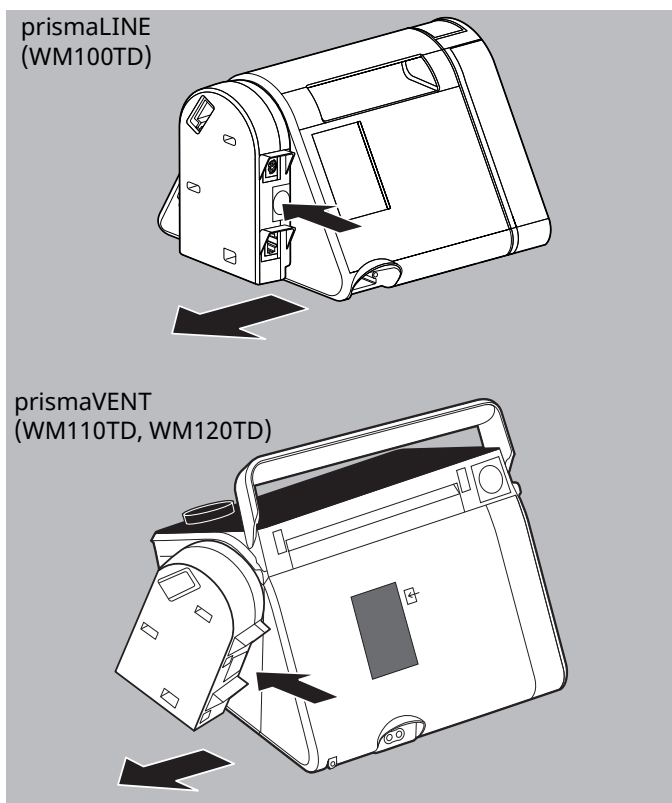2. Remove the module sideways from the therapy device.
3. Seal off the system interface on the therapy device again using the cover.

*Result* The prismaCONNECT module has been removed.  
The **C** symbol disappears from the display of the therapy device.

## 4.3 Configuring the prismaCONNECT module

### 4.3.1 prismaLINE (WM100TD)

- Requirement*
- The prismaCONNECT module is connected (see "4.1 Connecting the prismaCONNECT module", page 39).
  - The therapy device is switched on and is in **Standby** mode (see instructions for use of the therapy device).
1. Select the network settings on the therapy device: *Expert sector | Main Menu | Device | Network* (see instructions **Additional information for hospital personnel**).
  2. If required, select **Type** (default setting is **DHCP**).
 

**DHCP:** The prismaCONNECT module is assigned a dynamic IP address. If there is no DHCP server available, the IP address is assigned via zeroconf/Auto-IP in the range from 169.254.1.0 to 169.254.254.255.

**Manual:** configure the static IP address and a subnet mask.

It is also optionally possible to set a gateway and DNS.

*Result* prismaCONNECT is configured.  
The settings are saved in the prismaCONNECT module.

### 4.3.2 prismaVENT (WM110TD, WM120TD)

- Requirement*
- The prismaCONNECT module is connected (see "4.1 Connecting the prismaCONNECT module", page 39).
  - The therapy device is switched on and is in **Standby** mode (see instructions for use of the therapy device).
1. Select the network settings on the therapy device: *Expert sector | Main Menu | Device | Network*.
  2. Select **DHCP On** or **Off**.

**DHCP on:** The prismaCONNECT module is assigned a dynamic IP address. If there is no DHCP server available, the IP address is assigned via zeroconf/Auto-IP in the range from 169.254.1.0 to 169.254.254.255.

**DHCP off:** configure the static IP address and a subnet mask.

*Result* prismaCONNECT is configured.  
The settings are saved in the prismaCONNECT module.

## 4.4 Integrating the prismaCONNECT module into an IT network

You can control the therapy device and transfer data from the therapy device to a PC remotely via the IT network.

The prismaCONNECT module can be integrated into an IT network to create a connection between the therapy device and a PC. The network must provide the following properties:

- UDP and TCP ports 51337 and 51338 must be enabled and must not be blocked by a firewall.
- The UDP protocol is required for automatic identification of the devices in the network; data exchange then takes place via the TCP protocol.

If the IT network fails, you can continue to operate the therapy device via the display and transfer data via the SD card or the USB port. This means that failure of the IT network does not result in hazardous situations.

# 5 Hygiene treatment

## 5.1 General information

- Wear appropriate safety gear for disinfection.
- Follow the instructions for use of the disinfectant used.

## 5.2 Intervals

Interval	Action
Weekly	Clean modules
If required, e.g., following infectious diseases or severe contamination	Disinfect modules

## 5.3 Subjecting module to a hygiene treatment

### NOTICE

#### Material damage due to ingress of liquids!

Ingress of liquids may damage therapy device, components, and accessories.

⇒ Do not immerse therapy device, components, and accessories in liquids.

*Requirement* The therapy device is switched off (see instructions for use of the therapy device).

1. Disconnect the module from the therapy device ([see "4.2 Removing the prismaCONNECT module", page 41](#)).
2. Subject the module to a hygiene treatment in accordance with the table below:

Part	Cleaning	Disinfection	Sterilization
Housing	Wipe down: Use water or mild detergent	Disinfect by wiping (Recommended: terralin® protect or perform advanced Alcohol EP)	Not permitted
Network cable			

3. Leave the module housing and in particular the electrical interfaces to dry thoroughly.
4. Connect the module to the therapy device (see "4.1 Connecting the prismaCONNECT module", page 39).
5. Switch on the therapy device (see instructions for use of the therapy device).

*Result* The module has been subjected to hygiene treatment.

# 6 Function check

## 6.1 Intervals

Perform a function check at regular intervals:

- After every hygiene treatment
- After every repair
- At least every 6 months

## 6.2 Performing a function check

1. Check whether the module has been set up and connected properly in accordance with the instructions for use.
2. Check housing for cracks, damage, and severe soiling.
3. Connect the module to the device (see ["4.1 Connecting the prismaCONNECT module"](#), page 39).
4. Check whether the symbol  for prismaCONNECT is shown in the display of the therapy device.
5. Connect the Ethernet cable to the LAN port and to a PC or an IT network.
6. Check whether the network symbol  is shown in the display of the therapy device.
7. If the module is not working properly or is damaged: Contact your specialist dealer.

*Result* The function check is complete.

# 7 Faults

If you are unable to eliminate faults with the aid of the table, contact your specialist dealer to have the device repaired. To avoid exacerbating the damage, do not continue operating the device.

Fault	Cause	Remedy
 symbol does not appear in the display of the therapy device.	Contact problem or defective prismaCONNECT module.	Remove prismaCONNECT and reconnect it (see 4.2, p. 41), (see 4.1, p. 39). If the fault persists: Have prismaCONNECT module replaced.
 symbol does not appear in the display of the therapy device.	Ethernet cable defective or not connected.	Replace Ethernet cable.
No therapy device found in the prismaTS software.	IP address configured incorrectly.	Check the current IP address on the display of the therapy device: If the IP address is not correct, correct the settings (see 4.3, p. 42).
	Firewall blocking packets.	Enable UDP and TCP ports 51337 and 51338.

## 8 Servicing

The prismaCONNECT module is designed for a service life of 6 years.

If used in accordance with intended purpose in line with these instructions for use, the prismaCONNECT module requires no maintenance.

If you find defective parts during the function check ([see "6 Function check", page 46](#)), contact your specialist dealer.

# 9 Transport, storage and disposal

## 9.1 Transport and storage

Transport and store the device under the specified ambient conditions ([see "10.1 Technical specifications", page 50](#)).

## 9.2 Disposal

### 9.2.1 Electronic scrap

Do not dispose of the product in domestic waste. To dispose of it properly, contact a licensed, certified electronic waste disposal merchant. This address is available from your Environment Officer or from your local authority. The device packaging (cardboard and inserts) can be disposed of in paper recycling facilities.

# 10 Annex

## 10.1 Technical specifications

Specification	prismaCONNECT
Product class to MDR (EU) 2017/745	I
Dimensions W x H x D in cm	4.3 x 12.8 x 8
Weight in kg	0.185
Temperature range Operation Transport and storage	+5 °C to +40 °C -25 °C to +70 °C
Permitted humidity for operation, transport, and storage	Relative humidity 15 % to 93 %, no condensation
Air pressure range	700 hPa to 1060 hPa, corresponds to an altitude of approx. 3000 m above mean sea level
Electrical rating	12 V DC
Power capacity	1.5 VA*
Ethernet port insulation	4 kV
Classification to IEC 60601-11: Type of protection against electric shock	Protection class II
Protection against damaging ingress of water and solids	IP21
Classification to IEC 60601-1: Operating mode	Continuous duty
Electromagnetic compatibility (EMC) to IEC 60601-1-2* Emissions Immunity	CISPR 11 Class A, IEC 61000-3 Parts 2 and 3 IEC 61000-4 Parts 2 to 6, 8, 11, 39

\*Data apply only in combination with the approved therapy device.

The right to make design modifications is reserved.

## 10.2 Scope of delivery

Part	Article number
prismaCONNECT basic device	WM 29671
Instructions for use	WM 67940

## 10.3 Accessories

Part	Article number
Network cable, 3 m, CAT-6	WM 98273

## 10.4 Spare parts

You can order spare parts separately if necessary. A current list of spare parts can be obtained on the manufacturer's website or through your specialist dealer.

## 10.5 Warranty

Löwenstein Medical Technology gives the customer a limited manufacturer warranty on a new original Löwenstein Medical Technology product and on any spare part fitted by Löwenstein Medical Technology in accordance with the warranty conditions applicable to the product in question and in accordance with the warranty periods from date of purchase listed below. The warranty conditions are available on the manufacturer's website. We will also send you the warranty conditions on request. In the event of a claim under warranty, contact your specialist dealer.

Product	Warranty periods
Löwenstein Medical Technology devices including accessories (except: Masks)	2 years
Masks including accessories, batteries (unless quoted differently in the technical documentation), sensors, circuits	6 months
Disposable products	None

## 10.6 Declaration of Conformity

Löwenstein Medical Technology GmbH + Co. KG, Kronsaalsweg 40, 22525 Hamburg, Germany, the manufacturer of the devices described in these instructions for use, hereby declares that the product complies with the relevant provisions of Medical Devices Regulation (EU) 2017/745.

The unabridged text of the Declaration of Conformity can be found on the manufacturer's website.

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