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Mouthpiece Ventilation Set Instructions for Use

1 Intended use

The Mouthpiece Ventilation Set (MPV) is used to move the patient's mouthpiece into the desired position and hold it there. The flexible hose holder can be secured to a wheelchair or a bed, for example.

2 Safety

Please read these instructions for use carefully. They form part of the set described, and must be available at all times. Use the set for the designated purpose only (see section 1 "Intended use").


Please observe section 5 "Hygienic preparation" in order to avoid infection or bacterial contamination.

Also observe the respective instructions for use for the mouthpiece and the therapy device.

2.1 Symbols




Symbols on the type plate	Description	Symbols on the packaging	Description
	Consult instructions for use		Store between -25°C and +70°C
	CE mark (conformity of the product with the applicable European directives)		Permissible humidity for storage: 10% to 95% rel. humidity
	Manufacturer and date of manufacture		Protect the set against moisture
			Fragile, handle with care

3 Description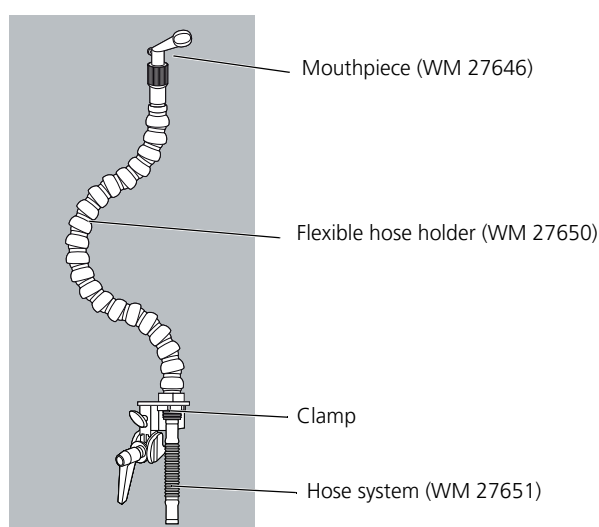



4 Assembly

4.1 Securing the hose system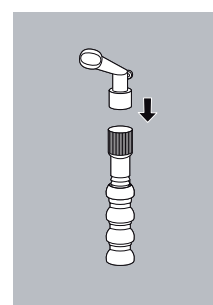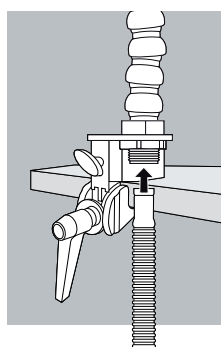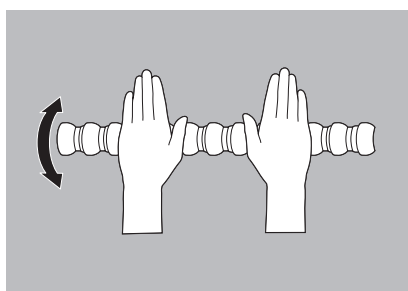


1. Form the hose holder into a straight shape, e.g., by pressing and rolling the hose holder on a flat, hard surface.
2. Insert the hose system into the hose holder
3. Secure the mouthpiece from underneath.

4.2 Securing the clamp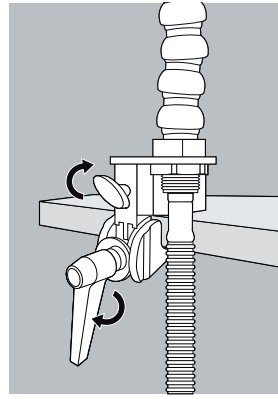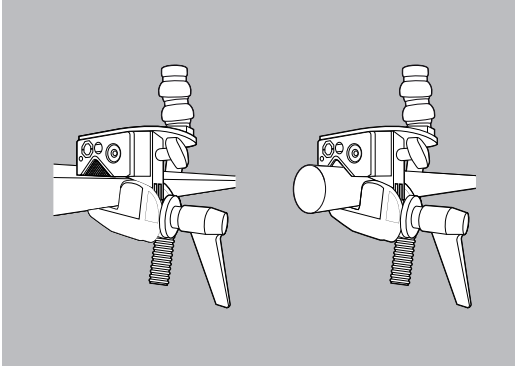

1. Attach the clamp to a flat surface.
or
Attach the clamp to a round object (e.g., a bar).
To do this, remove the triangular insert of the clamp.
2. To fix the clamp, turn the clamp lever clockwise.
3. Tighten the screw.

4.3 Adjusting the length of the hose

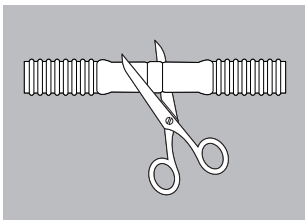1. To shorten the hose, cut it with a pair of scissors.
Note: Only cut the hose at the prepared points.

5 Hygienic preparation

Wipe all parts of the set weekly with a damp cloth if necessary: Use water or mild soap.
When changing patients: Use a new, unused set.

6 Maintenance

If the Mouthpiece Ventilation Set is used as intended in accordance with these instructions for use, it does not require any maintenance.

7 Replacement parts

A current list of accessories and replacement parts can be ordered on the internet site of the manufacturer or through your authorized specialist dealer.

8 Technical data

Specification	Flexible hose holder	Hose
Classification in accordance with Directive 93/42/EEC	I	IIa
Dimensions W x H x D in mm	45 x 720 x 85	27 x 1800
Weight in g	730	55

9 Warranty

Löwenstein Medical gives the customer a limited manufacturer warranty on a new genuine Löwenstein Medical product and on any replacement part fitted by Löwenstein Medical in accordance with the warranty conditions applicable to the product in question and in accordance with the warranty periods from date of purchase listed below. The warranty conditions are available on the website of the manufacturer. We will also send you the warranty conditions on request.

In the event of a claim under warranty, contact your specialist dealer.

Product	Warranty periods
Devices, incl. accessories (excluding: masks)	2 years
Masks, incl. accessories, batteries (unless otherwise stated in the technical documentation), sensors, hose systems	6 months
Disposable products	None

10 Declaration of conformity

Löwenstein Medical Technology GmbH + Co. KG, Kronsaaßweg 40, 22525 Hamburg, Germany, the manufacturer of the flexible hose holder described in these Instructions for Use, hereby declares that the product complies with the respective regulations of Medical Devices Directive 93/42/EEC. The unabridged text of the Declaration of Conformity can be found on the manufacturer's website.

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