

## Hygiene recommendations

# Sonata / Scala / Samoa



Item	Disinfectant	Application
<b>Effort sensors</b> (Pressure pads) <b>Sonata, Scala, Samoa main modules</b> (Protect contacts!) <b>Sonata Neuro module</b> (Protect contacts!) <b>Carrying console patient recorder</b> <b>Carrying console Neuro module</b> <b>Sonata Base</b> <b>Connecting cable Sonata / Scala to Sonata Base</b> <b>Bag with shoulder strap</b>	<b>mikrozid® universal wipes premium</b> Schülke & Mayr GmbH 	<b>Wipe disinfection</b> (Note acting time)  Make sure that no liquids get into the recorder or the plug connectors/connections! Please use the supplied plugs from the cleaning set for this purpose.

Order no.: 109186

Item	Disinfectant	Application
<b>All straps</b> <b>Thermistor</b> <b>Sonata Intercom cable</b> <b>USB cable for data transmission to PC</b> <b>Finger sensor SpO<sub>2</sub></b> <b>Snoring microphone</b> <b>Actimeter</b> <b>External position sensor</b> <b>ECG connecting cable</b>	<b>mikrozid® AF wipes</b> Schülke & Mayr GmbH 	<b>Wipe disinfection</b> (Note acting time)  Make sure that no liquids get into the recorder or the plug connectors/connections! Please use the supplied plugs from the cleaning set for this purpose.

Order no.: 109203

\* More detailed information can be found in the hygiene plan of the manufacturer Dr. Fenyves und Gut Deutschland.



  
 Dr. Fenyves und Gut Deutschland  
 Daimlerstraße 23  
 72414 Rangendingen, Germany

**Sales + Service**  
 Löwenstein Medical  
 Arzbacher Straße 80  
 56130 Bad Ems, Germany  
 T. +49 2603 9600-0  
 F. +49 2603 9600-50  
 info@loewensteinmedical.com  
 loewensteinmedical.com



© Protected by copyright. Reproduction of any kind requires the express consent of Löwenstein Medical.  
 All details subject to correction due to the printing process.

