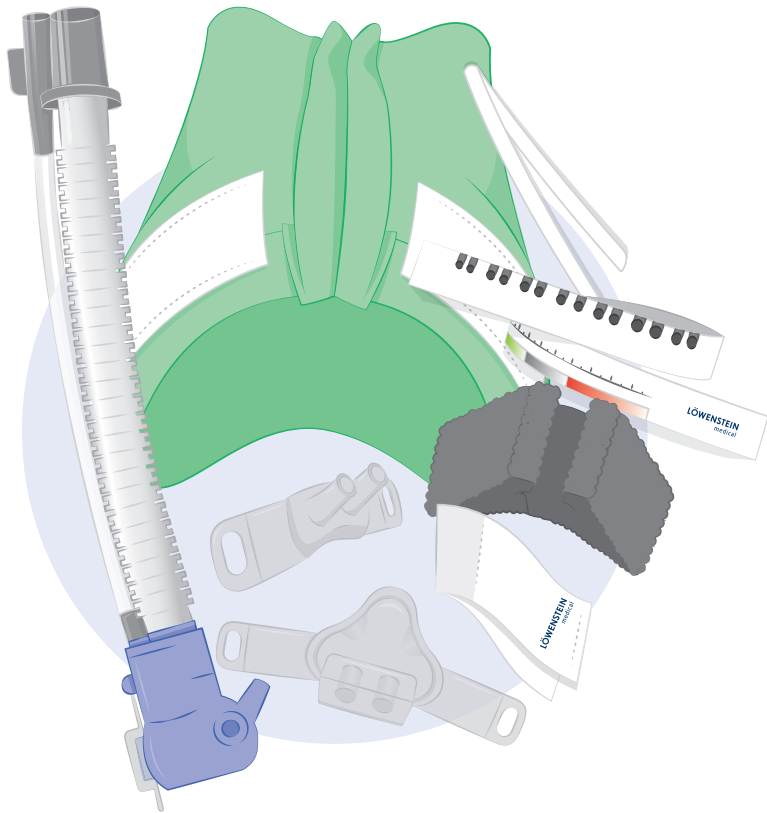
# Neojet® and accessories

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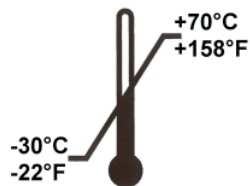
**Warning: Manual and use**

Before using the products, read these instructions for use carefully and follow the instructions precisely.

The Neojet must only be used with detailed knowledge of these instructions for use and by following them - and only for the purpose described in "Intended purpose".

Neojet must only be used in combination with an appropriate CPAP driver or ventilator and with suitable patient monitoring. Observe the warnings and the notices.

Use of devices which can be run in combination with Neojet (e.g.: CPAP driver or ventilator and humidifier) is described in the respective instructions for use for the devices. These instructions for use must likewise be followed.

**1. Intended purpose****1.1. Neojet NJ2000**

The Neojet NJ2000 is a disposable product for nCPAP therapy/non-invasive ventilation therapy for premature babies and neonates. The Neojet forms the interface to the patient. It is used in combination with a CPAP driver or a ventilator to supply the Neojet with the necessary gas.

**1.2. Bonnets, masks, prongs**

The bonnets, masks and prongs are products for use for nCPAP therapy or non-invasive ventilation of premature babies and neonates. They should be used in combination with Neojet nCPAP generators. The bonnets are available in various sizes and are used to secure the nCPAP generators and the masks and prongs in position. The masks and prongs are disposable silicone adapters which create the contact between the nCPAP system and the patient. The different sizes enable the gentlest and most leaktight possible connection to be made to the patient's nose.

**Warning: Using the Neojet**

- For clinical use only
- The patient's blood oxygen saturation must be monitored permanently during use.
- The product must only be used by trained staff.
- Neojet NJ2000, masks, disposable bonnets and prongs are designed to be used only once on one patient.
- The bonnets, masks and prongs must only be used in combination with a Neojet.
- Neojet must only be used in combination with a CPAP driver or ventilator.

**2. Functional principle and technical specification****2.1. Neojet**

The Neojet is a patient interface which can be used in combination with a CPAP driver or in CPAP mode with certain ventilators. The Neojet is connected to the CPAP driver or ventilator by means of the ventilation tube. The parameters are set on the CPAP driver or ventilator. The CPAP driver or the ventilator provides the necessary gas flow. The prong mask adapter can be tilted on the disposable Neojet NJ2000 product. This allows the direction of the Neojet to be adapted to suit the anatomy of the baby's nose, making it easier to secure it in position.

A silicone prong or a silicone mask selected to suit the size of the baby is attached to the prong mask adapter. The prong or mask ensures gentle contact between the CPAP equipment and the baby's nose.

The Neojet itself is attached to the bonnet the baby is wearing.

**Technical specifications:**

Flow connection:	M10/F7.4	Pressure connection:	F4.3
Weight:	10 g	nCPAP pressure:	≥ 4.5 cm H <sub>2</sub> O at 7L/min. flow

**2.2. Bonnets, masks, prongs**

A split piece of fleece fabric is sewn to the top of the polyamide bonnets. A foam wedge (Neofoam for Neojet) is positioned between the fleece parts, the Neojet is put inside and is secured with a separate hook and loop retaining strap. The additional straps included with the product are used to stabilize masks or prong position and their hook and loop ends allow them to be positioned anywhere on the bonnet fabric.

The masks and prongs are made of 100 % silicone. They each contain eyes for attaching the straps to the bonnet. They also include a pair of holes for connecting to the Neojet; the tilting adapter of the Neojet is pushed into these holes.

	Disposable bonnets	Masks	Prongs
Material	Polyamide	Silicone	Silicone
Use	Disposable product	Disposable product	Disposable product
	 		
Available sizes	XXS to XXXL	XS to XL	XS to XL
<b>nCPAP generators which can be combined:</b> Neojet NJ2000			

**3. Structure and use**

**Equipment required:**

In order to perform effective CPAP therapy with the Neojet, the following equipment is required:

- A CPAP driver or a ventilator which provides a connection for external measurement of CPAP pressure.
- An active humidifier.
- A Neojet.
- A heated patient circuit compatible with the humidifier and the Neojet is required. The connection at the end of the inspiration tube must be F10/M7.4 and the pressure measurement line must include a Luer connector (M4.3) in order for it to fit on the Neojet.
- A tape measure to determine the size of bonnets and prongs.
- A bonnet - select the correct size to fit the baby.
- A mask or prong of the correct size.

**Warning: Packaging**

If the packaging for a part is damaged or if the products themselves are damaged, these parts must not be used and must be disposed of.

You should also examine the holes in the generator and the prongs/masks to check they are clear. If the openings are not clear, the products affected must not be used.

Structure and setup of the system

- (1) Connect the tube set on the CPAP driver/ventilator to the humidifier. If necessary, use the enclosed adapter. Push the temperature sensors in up to the stop; their tip must be in the gas flow.
- (2) Connect the Neojet to the inspiration tube (larger connection on the Neojet). Push the pressure measurement line into the pressure measurement port of the Neojet (smaller connection) and connect the pressure measurement line to the pressure measurement port of the CPAP driver or ventilator.
- (3) Seal the prong adapter of the Neojet by hand and set the parameters, e.g. pressure, on the CPAP driver or ventilator. If this is not possible, check the tube connections for leaks. If all the connections are correct and the Neojet is still causing problems, it must be disposed of and not used.

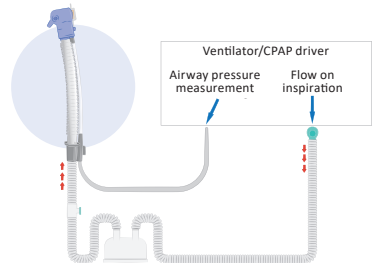
Figure 1

**Important note: Faulty Neojet**

If the Neojet is not working correctly, it must not be used.

- (4) Use the tape measure to help select a prong or a mask. The selected prong should be large enough to seal the whole of the nostril. The selected mask should fit exactly next to the wing of the nose. A prong is pushed onto the Neojet generator far enough for the curved side of the prong to point towards the patient's top lip. The mask is put on so that it fits the shape of the nose (Figure 4).

**Note: Prongs**

Always use the largest-possible prong to avoid leaks and to get the best possible fit - the lower the leakage, the higher and more stable the CPAP pressure.

**Warning: Prongs and masks**

Push the masks or prongs completely onto the Neojet using the holes provided for this purpose and do not use any cream in the vicinity of the prongs, masks or Neojets otherwise the prong or mask may slip off the Neojet (Figures 6 and 7).

The prongs and masks must not be reprocessed and used again. They are intended to be disposable. Reprocessed prongs and masks may harden and injure the patient.

- (5) Use the tape measure and appropriate color coding scheme to select a bonnet of the correct size (Figures 3 and 5). The bonnet should fit well, but not so tightly that it leads to deformation of the head. Remove the hook and loop tape from the bonnet and put it aside where you can reach it easily. The bonnet is placed on the head so that the hole between the two fleece parts is central above the forehead. The bonnet should cover the ears and be pulled down to the nape of the neck. Thread the securing straps through the eyes on the prong or mask (Figure 8).

**Note about bonnets:**

Pull the bonnet down to just above the patient's eyebrows. Bonnets which are too small will lead to deformation of the patient's head. Bonnets which are too big will not secure the CPAP generator adequately.

- (6) Select the most favorable angle for the prong adapter of the Neojet (45° or 60°). The sharper angle is usually better if you are using a prong, whereas with a mask, the shallower angle is better (Figures 6 and 7). Place the Neofoam foam wedge in the center of the bonnet between the two fleece parts. Fit the Neojet in the round opening provided for it in the foam. Guide the prong into the nostrils. Maintain a small distance between the septum and the base of the prong. When using the mask, ensure that it is contacting the skin evenly (Figure 8). Leave the nostrils free and do not cover with any part of the mask. Secure the Neojet to the fleece parts of the bonnet from the top using the hook and loop end of the strap. Secure the hook and loop ends of the straps in the area of the ears as an extension of the fixation loops of the prong. Do not over-tighten the straps, they are designed purely to provide sufficient hold from the sides (Figure 9).

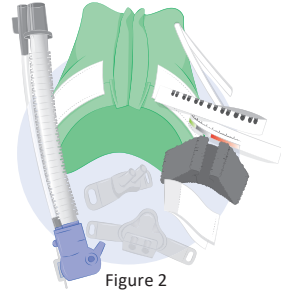
Figure 2

Figure 3

Figure 4

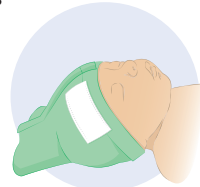
Figure 5

Figure 6

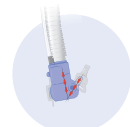
Figure 7

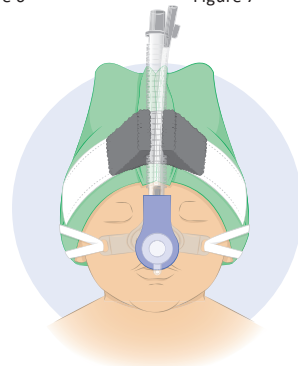
Figure 8

**Important note: Attaching the prongs and masks**

- Do not guide prongs all the way into the nostrils. The connecting block should not be in contact with the nose.
- Regularly switching between prongs and masks is an effective way of preventing or reducing damage to the patient's skin.
- Do not over-tighten the straps. The Neojet is held in position primarily by the bonnet and the foam. If leakage is too high, check the selected prong or mask size – tightening the straps will not necessarily fix the problem.
- Overtightening the straps or introducing the prongs too far can lead to pressure points on the baby's nose or skin.

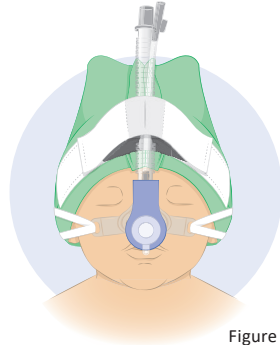
Figure 9

**Important note: Securing the tubes**

- Ensure that the weight of the tubes is not pulling directly on the patient's head and the interface. Rest the tubes on supports.
- The inspiration tube should be angled away from the patient and the Neojet so that condensation does not flow towards the baby (Figure 10).

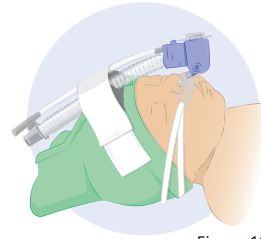
Figure 10

- (7) Regularly check CPAP settings and that the arrangement is secure.

**Important note: Regular check that arrangement is secure**

Regularly check the fit of the Neojet and its accessories and that they are secure, otherwise pressure points may form. Check the prongs and masks regularly for accumulation of secretions, otherwise blockages might occur.

**Note:**

If a humidifier is used, the CPAP pressure generated may rise due to material expansion under the effects of heat whilst flow remains the same.

**4. Ventilators which can be connected to the Neojet****4.1. Neojet**

The Neojet can be connected to a CPAP driver or a ventilator as long as the following conditions are met:

- Device delivers a controlled gas flow of 4 l/min to 15 l/min
- Device does not require exhalation gases to be returned to the device
- Device has a connection for external pressure measurement allowing the pressure generated in the Neojet to be measured
- Device works without an external flow sensor
- The device must have an M22 or F15-sized connection for inspiration and a Luer connection (4.3 mm internal diameter) for pressure measurement. It can then be connected to the tube sets or there must be tube sets (inspiration and pressure measurement) for the device which have an F10 or M7.4 connection on the Neojet side and a pressure measurement line with a Luer connector (4.3 mm external diameter).

**Among other devices, Neojet can be combined with:**

- CPAP drivers
- Leoni 1
- Leoni mobil
- Leoni 2
- Leoni 2 transport
- Leoni plus
- Leoni plus transport

**4.2. Bonnets, masks, prongs**

All bonnets, masks and prongs may only be combined with the Neojet nCPAP generator.

## 5. Accessories

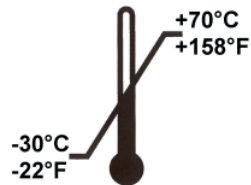
<b>Neojet</b> 20 pcs	NJ2000		
<b>Prongs</b> - 10 pcs disposable			
Micro	NJ1500-01	Large	NJ1500-22
Small	NJ1500-21	Large-wide	NJ1500-33
Medium	NJ1500-02	X large	NJ1500-03
Medium-wide	NJ1500-32		
<b>Masks</b> - 10 pcs disposable			
X Small	NJ1500-08	Medium	NJ1500-05
Small	NJ1500-04	Large	NJ1500-06
<b>Masks</b> - 5 pcs disposable			
X large	NJ1500-07/05		
<b>Bonnets</b> - 10 pcs disposable			
XX Small	NJ1513-10	Large	NJ1517-10
X Small	NJ1514-10	X large	NJ1518-10
Small	NJ1515-10	XX large	NJ1519-10
Medium	NJ1516-10	XXX large	NJ1520-10
<b>Neof foam</b> - 10 pcs disposable	NJ1030		
<b>Tubes</b> - 10 sets - disposable			
Disposable tube sets for Neojet for LM 2000 with humidifier chamber	n160nv1c		

### Warning: Accessories

The Neojet will only work in combination with these masks, prongs and bonnets, otherwise correct function cannot be guaranteed. **Bonnets, masks and prongs may only be combined with the Neojet nCPAP generators.**

## 6. Storage and disposal

The Neojet and its accessories must be stored in a clean, dry room. Temperatures between -30 °C and 70 °C have no negative impact. The Neojet and its accessories can be disposed of in normal domestic waste following use on non-infectious patients.

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## 5. Accessoires

Neojet, 20 pièces	NJ2000		
<b>Lunettes nasales, 10 pièces, usage unique</b>			
Micro	NJ1500-01	Large	NJ1500-22
Small	NJ1500-21	Large-wide	NJ1500-33
Medium	NJ1500-02	X large	NJ1500-03
Medium-wide	NJ1500-32		
<b>Masques, 10 pièces, usage unique</b>			
X Small	NJ1500-08	Medium	NJ1500-05
Small	NJ1500-04	Large	NJ1500-06
<b>Masques, 5 pièces, usage unique</b>			
X large	NJ1500-07/05		
<b>Bonnets, 10 pièces, usage unique</b>			
XX Small	NJ1513-10	Large	NJ1517-10
X Small	NJ1514-10	X large	NJ1518-10
Small	NJ1515-10	XX large	NJ1519-10
Medium	NJ1516-10	XXX large	NJ1520-10
Neof foam, 10 pièces, usage unique	NJ1030		
<b>Tuyaux - 10 circuits, usage unique</b>			
Circuits à usage unique pour Neojet, pour LM 2000 avec chambre d'humidification	n160nv1c		

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## 5. Accessori

Neojet 20 pezzi	NJ2000		
<b>Nasocannule - 10 pezzi monouso</b>			
Micro	NJ1500-01	Large	NJ1500-22
Small	NJ1500-21	Large-wide	NJ1500-33
Medium	NJ1500-02	X large	NJ1500-03
Medium-wide	NJ1500-32		
<b>Maschere - 10 pezzi monouso</b>			
X Small	NJ1500-08	Medium	NJ1500-05
Small	NJ1500-04	Large	NJ1500-06
<b>Maschere - 5 pezzi monouso</b>			
X large	NJ1500-07/05		
<b>Berretti - 10 pezzi monouso</b>			
XX Small	NJ1513-10	Large	NJ1517-10
X Small	NJ1514-10	X large	NJ1518-10
Small	NJ1515-10	XX large	NJ1519-10
Medium	NJ1516-10	XXX large	NJ1520-10
<b>Neofoam</b> - 10 pezzi monouso	NJ1030		
<b>Tubi - 10 set - monouso</b>			
Set di tubi monouso per Neojet per LM 2000 con camera dell'umidificatore	n160nv1c		

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*© Copyright 2020– All contents, especially texts, photographs and graphics are copyright protected.