**GE** Gebrauchsanweisung **EN** Instructions for Use  
**FR** Mode d'emploi **IT** Istruzioni d'uso

## Sonata / Scala

<b>Deutsch</b>	<b>3</b>
<b>English</b>	<b>19</b>
<b>Français</b>	<b>35</b>
<b>Italiano</b>	<b>51</b>

**Appendix A - Konformitätserklärung / Declaration of Conformity**

**Appendix B - Bestellinformationen**

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## Table of Contents

<b>1. Introduction.....</b>	<b>20</b>
1.1. Assigned Purpose.....	20
1.2. Contraindications.....	20
1.3. Accessories.....	20
<b>2. Information.....</b>	<b>21</b>
2.1. Safety-related information.....	21
2.2. General information.....	22
2.3. Technical inspection.....	22
2.4. Liability for functioning or damage.....	22
<b>3. Recording measurement data.....</b>	<b>23</b>
3.1. Sensors.....	23
3.1.1. Sensors for flow and snoring noises.....	23
3.1.2. Thermistor sensor for breathing.....	23
3.1.3. Sensor for PAP (pressure).....	23
3.1.4. Sensor for oxygen saturation and pulse frequency .....	23
3.1.5. Sensor for respiratory effort.....	23
3.1.6. Sensor for body position.....	23
3.1.7. Sensor for leg movement.....	24
3.1.8. Sensor for ECG.....	24
3.1.9. Sensor for Snoring (external).....	24
3.1.10. Sensors for EEG / EOG / EMG / EXG.....	24
3.1.11. Actimeter.....	24
3.2. Preparing for measurement (Ambulatory).....	24
3.3. Operating elements of the device.....	24
3.4. Attaching the device unit and starting the measurement.....	25
3.5. Online Measurements (Inpatient).....	25
<b>4. Service and Maintaining the device.....</b>	<b>25</b>
4.1. Charging the battery.....	25
4.1.1. Charging Cradle.....	25
4.1.2. Charger.....	25
4.1.3. Sonata Base.....	26
4.2. Cleaning instructions.....	26
4.3. Maintenance.....	26
4.4. Transport and storage conditions.....	26
<b>5. Evaluation software for PC.....</b>	<b>27</b>
5.1. Installing the software.....	27
5.2. Selecting a printer.....	27
<b>6. Troubleshooting.....</b>	<b>27</b>
<b>7. Technical specifications.....</b>	<b>28</b>
<b>8. Used Symbols.....</b>	<b>29</b>
<b>9. EMC Information.....</b>	<b>30</b>
<b>10. Index.....</b>	<b>33</b>

## 1. Introduction

### 1.1. Assigned Purpose

The Sleep Diagnosis Systems Sonata and Scala are recording systems for use in professional health care facility and home healthcare environment. The devices are used for differentiated pre-diagnosis (Scala) or diagnosis (Sonata) of sleep disorders and therapy control. It is intended for use by or on the order of physicians in adults and children.

The devices allow the continuous recording of up to 24 (Scala) or 49 (Sonata) measurement channels for at least 15 hours without data reduction. The following physiological variables are recorded:

- Flow
- Thermistor
- Oxygen saturation SpO<sub>2</sub>
- Pulse frequency
- Pulse wave
- Body position
- Snoring sounds (internal microphone)
- Snoring sound (external microphone)
- PAP (Positive Airway Pressure)
- Ambient Light
- Thorax effort
- Abdomen effort
- 4 x EXG

only Sonata:

- 6 x ECG
- Central heart Frequency
- PTT (Pulse Transit Time)
- Systolic Blood Pressure Trend
- 2 x EMG (Leg movement)
- 3 x EMG (Chin)
- 6 x EEG
- 2 x EOG
- 2 x Actimeter
- External position sensor

optional (Sonata and Scala):

- 8 x Analogue channel

### 1.2. Contraindications

There are no absolute or relative contraindications to using the device. In the following cases, the device must be used under the supervision of qualified medical personnel:

- in patients with acute life-threatening illnesses.
- in patients with acute, serious infections.
- in mentally confused patients.
- in infants and children

### 1.3. Accessories

The device system as a whole (ME system) contains four components:

- The microprocessor controlled Sonata / Scala recording device (patient recorder)
- The Neuro module
- PC software for measurement data presentation and PC analysis

- Optional: Sonata base for 8 analogue inputs

It is also equipped with the following applied parts:

- SpO<sub>2</sub> finger sensor with cable for obtaining pulse frequency and oxygen saturation values
- Flow prong for obtaining the respiratory signal.
- Adapter hose for measurement during PAP Therapy
- Flexible carry belt with integrated pressure transducers (effort sensor) to fasten the device to the patient
- Flexible belt with integrated pressure transducers (effort sensor) to record abdominal effort
- carrying console for patient recorder and Neuro module.
- Shoulder bag for storing the Scala

Other accessories:

- USB Interface cable for data transfer between the patient recorder and PC
- Connection cable between the patient recorder and the Neuro module
- Battery cradle

optional applied parts:

- Thermistor for additional breathing detection
- External snoring microphone
- Actimeter (only Sonata)

Other optional accessories:

- External position sensor (only Sonata)
- 2-pol. and 3-pol. ECG cable
- Extension EMG cable Leg
- Charger
- Differential pressure adapter for measurement under PAP ventilation
- Thorax sensor RIP
- Abdomen sensor RIP

The PC is not suitable for a domestic or patient environment due to lack of drip protection!

The data can be displayed and analysed on a standard commercial PC. The measurement curves and the analysis results can be printed on all commonly available printers, including dot-matrix, laser or ink jet printers.

The PC software analysis program has the following minimum configuration requirements

- PC with Windows operating system and i5 processor (i7 processor recommended)
- Microsoft Windows 10 or higher
- Dedicated graphic adapter
- 16 GB of RAM (32 GB recommended)
- 500 GB of disk space on the hard drive
- Mouse
- Free USB port
- Printer with Windows driver

## 2. Information

### 2.1. Safety-related information

#### Observe the instruction manual:

Every use of the device requires exact knowledge and observance of this instruction manual. The device is only intended for the use described herein.

#### No alarm function available!

The device is not suitable for continuous monitoring of vital or physiological functions (for example, intensive monitoring, monitoring operation) since there is no SpO<sub>2</sub> alarm. No direct data analysis takes place in the device.

#### No SIDS Monitoring:

The device is not suitable for use as a SIDS monitor (SIDS: Sudden Infant Death Syndrome, sudden child death).

#### Patient instruction:

Patient instruction must be provided by a healthcare professional trained in the device. An attached short manual does not replace the manual instructions or the warning of possible hazards.

#### Separation from the supply network::

To isolate the device from the supply network, the plug of the power supply must be disconnected.

#### **Do not open the device!**

#### **Warning:**

Additional devices that are connected to medical electrical (ME) devices must be able to meet their IEC or ISO standards. Anyone connecting additional devices to ME devices is a system configurator and is responsible for the ME system being in compliance with the normative requirements (for example, IEC 60601-1).

#### **Warning:**

For online measuring you must use an electrical or optical isolator for connection of the device to the PC (available from your home care provider). A connection of the device to the PC without electrical or optical isolator is only allowed if all patient connections have been removed first. Only Physicians or trained staff is allowed to perform online measurements.

#### **Warning:**

Magnetic and electrical fields can impact on the functioning of the device. Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally. When operating the device, ensure that all third-party devices being operated in the vicinity are meeting their relevant EMC requirements. X-Ray equipment, HF surgical devices, tomography devices etc. can interfere with

other devices as they may emit higher levels of electromagnetic interference.

#### **Warning:**

The device has no defibrillator protected application parts! Disconnect the device before defibrillation! A direct application of the device to the heart (esp. ECG!) is not allowed!

While the device is being worn, no invasive or intracorporal measurements and interventions (e.g., electro- or HF-surgical devices) may take place.

#### **Warning:**

Using multiple devices for patient screening could result in excessive leakage current above the allowed value!

#### **Warning:**

When using the device on patients with active implants, such as Cardiac pacemakers, peripheral nerve stimulation devices, tongue pacemaker etc., medical professionals should be alert to possible interference with the device or the implant.

#### **Warning:**

Applying the device to more than one patient at any one time is not allowed.

#### **Warning:**

The device must not be operated in vehicles and aircraft.

#### **Warning:**

When applying sensors to the patient, please ensure that no electrical line of the device is in contact with any other electrical conductive parts including ground.

#### **Warning:**

Avoid placing this instrument in direct sunlight or in close proximity to intense heat. Prevent also the contact with dust, lint, dirt, moisture and liquids.

#### **Warning:**

Children or incompetent persons should not use this device unattended without first having obtained instructions for the safe use of the device. Ensure that infants, children and animals cannot touch the device unattended

#### **Warning:**

Caution must be taken to ensure that cables do not encircle the patients neck. Special attention is needed in the case of children.

#### **Warning:**

Before the battery is charged, the charger and the battery must be checked for external damage!

#### **Warning:**

Before every application, the housing of the device and Neuro module as well as the cables and sensors must be checked for external damage!

#### **Warning:**

The device and PC software are not intended for automatic diagnosis only. The measurement data

must be always reviewed and evaluated manually by a qualified physician or healthcare professionals.

## 2.2. General information

This instruction manual should be regarded as a component of the device. It should be kept on hand somewhere near the device at all times. Reading and understanding the instruction manual is a prerequisite for proper use and correct handling of the device in order to maintain the safety of the patient and operator.

The guarantee is valid for a period of 48 months for the device and 9 months for the equipment from the date of sale.

Only accessories which are listed in this instruction manual should be used with the device. We cannot guarantee the safe operation and function of the device if unknown proprietary accessories / consumables are used that have not been tested. (E.g. Patient leads, Sensors, Consumables etc.)

**Damages resulting from the use of third-party accessories or consumables shall render this warranty void.**

The manufacturer will only assume responsibility for the device in terms of safety, reliability and function if:

- 1. assembly, add-ons, reinstallation, changes and repairs are carried out by the manufacturer or a qualified agent authorized by the manufacturer to do so;**
- 2. The device is used in accordance with the instruction manual.**

All printed material relates to the model of the device and the safety regulations at the time of printing. All devices, switches, processors, software programs and names contained herein are subject to copyright law.

The manufacturer shall only be liable for the malfunction of the device and its software if used in the normal operating conditions in accordance to this manual.

Medical devices must only be operated by a qualified experienced person(s) to ensure correct handling of such devices.

The operator must read and understand the user manual to operate the device correctly.

The operator must check the functionality of the device before each use to ensure it is in sound condition and in good working order.

The user must report any serious incident related to the device to the manufacturer and to the competent authority of the Member State in which the user is established.

Functional testing of the device must be carried out at regular intervals. It is recommended that this is conducted once a month.

Upon reaching the end of its service life, the device and its accessories should be disposed of in accordance with the WEEE Directives or relevant Electronic Disposal protocol.

For further details please contact the manufacturer.

## 2.3. Technical inspection

The device is maintenance-free. It is recommended to have a technical inspection carried out by the manufacturer or an agency authorized by the manufacturer every 24 months.

A functional tester can not be used to assess the accuracy of the finger sensor or pulse oximeter of the device.

The following tests need to be carried out by the operator before each measurement:

- Visual inspection of the device and accessories for obvious damage which could result in mechanical failure.
- Testing all hose connections are air tight.
- Checking the display LEDs.
- Testing the battery capacity.

## 2.4. Liability for functioning or damage

In the event of improper use or repair by the owner or operator, the liability for the device will be transferred to those parties. The manufacturer shall not be liable for damages arising from non-observance of this instruction manual. The actual guarantee and warranty conditions in the manufacturers terms of sale/delivery are not extendable.

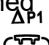
### 3. Recording measurement data

#### 3.1. Sensors

##### 3.1.1. Sensors for flow and snoring noises

The detection of patient flow is determined by the use of a nasal flow prong that transmits the pressure signal to an internal pressure transducer of the device. The flow prong can be easily applied by the user and will have no adverse affect on the quality of sleep. The use of a flow prong also eradicates the problem where adhesive sensors cannot be used. (i.e.: patients with beards). In addition the flow prong is an economical choice of sensor as it is commonly available. No additional sensor is required for respiratory and snoring detection. Respiratory sounds are transmitted to the device via the flow prong. The built in microphone detects the sounds which are then electronically analysed. Therefore additional microphones are not required.

Due to the high sensitivity of the internal pressure transducer, the device is capable of measuring and recording ultra fine pressure differences associated with patients who mouth breathe.

The flow prong must be attached to the socket marked with the following symbol: 

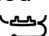
The flow prong should be used in accordance with the manufacturer's instructions.

#### Warning:

The flow prong is intended for single patient use. No attempt should be made to clean or use the flow prong the more than once. Multiple use can result in cross infection!

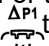
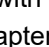
##### 3.1.2. Thermistor sensor for breathing

Additional to the flow prong a Thermistor can be used to measure the breathing. Please also observe the instructions accompanying the sensor.

The Thermistor must be attached to the socket marked with the following symbol: 

##### 3.1.3. Sensor for PAP (pressure)

When performing measurements on a patient undergoing PAP therapy, a Pressure Diff Adapter (PDA, two connections) or an adapter hose (one connection) is used instead of the flow prong.

PDA connection: The patient-near connection of the PDA is connected with the socket marked with  the patient-far connection with the socket marked with 

Connection of PAP adapter: The PAP adapter is connected with the socket marked with .

##### 3.1.4. Sensor for oxygen saturation and pulse frequency

A pulse oximeter has been integrated into the device for the purpose of measuring oxygen saturation and pulse frequency. A failure of the finger sensor or a

missing data update by the pulse oximeter is indicated to the user by the red LED on the device. At the same time the values for SpO<sub>2</sub> and pulse are fed to 0.

Please ensure that blood circulation at the point of measurement is not affected by the manner in which the sensor has been secured. Excessive pressure should not be exerted on the finger, especially if the temperature exceeds 41 ° C.

When using a finger sensor please secure the sensor and cable. All nail polish (coloured or clear) must be removed from the measuring finger to obtain usable data.

To minimize interference (e.g., motion artifacts), the pulse oximetry values are filtered by digital data processing. As a result of data averaging and signal processing, slight delays in the display of the pulse oximetric values occur. The internal pulse oximeter works with a time horizon of 4 seconds. By additionally considering the change tendency, the minimum value of saturation at the end of an apnea is correctly reproduced. The data is updated at every beat, so there are no measurable delays due to data refresh and transmission.

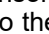
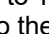
The finger sensor must be attached to the socket marked with the following symbol: 

Please read and understand the instructions accompanying the finger sensor.

##### 3.1.5. Sensor for respiratory effort

The sensors used for recording thorax and abdominal movement consists of two small rubber pressure pads that are connected to the device via measuring hoses. The sensors are inserted into the pockets of the elasticated effort belts. The thorax belt is applied at the height of the sternum, the abdominal belt in the stomach region.

For hygiene reasons and to avoid allergic reactions the belts should be **worn over the clothes**.

The effort sensor to monitor thorax movement must be attached to the socket marked with  and the effort sensor to monitor abdominal movement must be attached to the socket marked with .

The effort belts are adjustable in size due to the Velcro fasteners, and should be suitable for most patients. These belts are however also available in special sizes.

#### Note:

Over stretching of the belts may result in poor signals and possible loss of data.

Thorax and Abdomen sensors with RIP technology can be uses alternatively. Please read and understand the instructions accompanying this sensors.

##### 3.1.6. Sensor for body position

The position sensor integrated in the device displays information related to the patient's body position during the study.

For accurate determination of body position, ensure the device is applied to the patient correctly. The body positions detected are: Supine, Prone, Left side, Right side, Upright.

An external position sensor can be used alternatively for the Sonata by attaching it with the socket marked with 

### 3.1.7. Sensor for leg movement

To diagnose restless or periodic leg movements (restless leg), the Sonata device can be equipped with EMG electrodes. The device then enables continuous recording of leg movements and associated analysis results in the report.

#### Attaching the leg sensor:

All commonly available types of electrodes with 1.5mm safety plugs can be used for both bipolar EMG channels. The sockets for the EMG electrodes are marked with "LEG".

For application and cleaning of the electrodes please refer to the manual of the manufacturer.

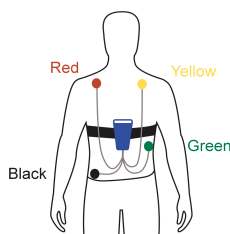
For long time electrodes a very careful preparation of the skin is necessary!

### 3.1.8. Sensor for ECG

For ECG all commonly available adhesive electrodes can be used as consumables.

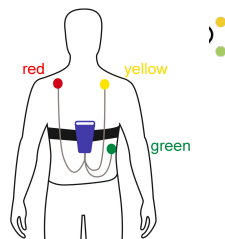
#### Attaching the ECG electrodes:

Scala:

The electrodes can be attached to the sockets marked with EXG.

Sonata:

The electrodes can be attached to the sockets marked with ECG.

**Important:** The ECG is not suitable for a differential cardiac diagnosis!

### 3.1.9. Sensor for Snoring (external)

Additionally to the internal snoring microphone an external microphone can be used to obtain the snoring sound. For adapting the snoring microphone please refer to the manual of the sensor.

The microphone must be attached to the socket marked with the following symbol: 

### 3.1.10. Sensors for EEG / EOG / EMG / EXG

For the neurological channels (EEG, EOG, EMG, EXG) all commonly available types of electrodes with 1.5mm safety plugs can be used.

For application and cleaning of the electrodes please refer to the manufacturers instructions.

**Note:** In order to guarantee good adhesion of the electrodes and strong frequency signals, clean the desired skin area with a sterile solution and apply a good quality conductive cream.

### 3.1.11. Actimeter

To record the movement activity of the patient activity sensors can be attached to the Sonata sockets marked with the symbol: 

Please read and understand the instructions accompanying the actimeter.

## 3.2. Preparing for measurement (Ambulatory)

In preparation of an ambulatory study where the patient will use the device at home, please ensure the following:

1. Charge the battery: see (Page 25).
2. Connect the device with the USB interface cable.
3. Start the PC software.
4. Fill out the fields relating to personal data of the patient in the menu item "Record / Initialize device" and start the transfer.
5. Connect the device with the USB interface cable. For testing purposes, both LEDs will light up initially. The red LED will then go off. The green LED on the device will remain lit for the duration of the initialization process.
6. A dialogue window will be displayed in order for the operator to determine the start time for recording.
7. The device is now ready for use. The device can be disconnected from the PC.

**Note:** remove cable by pulling on the plug - not on the cable.

## 3.3. Operating elements of the device

The device will automatically switch on / off when connected / disconnected to a PC. The device has an internal clock for timer recording. If the timer has been programmed, recording will start automatically at the predetermined time. Up to eight recordings can be programmed.

Note: If necessary the device can be switched on by the patient before the selected start time by pressing and holding the Start button for one second. By pressing the start button (longer than 7 seconds), the device can be switched off by the patient.

The charging condition of the units battery can be displayed by pressing the button on the battery display on top of the device: 

Attention: This function will only work when the device is switched off and disconnected from the PC.

### 3.4. Attaching the device unit and starting the measurement

When applying the sensors, make sure that they are applied in such a way that strangulation is not possible due to their length (for example flow prongs, finger sensor). In order to ensure reliable recording of data, you need to observe the following points when attaching the device and sensors. Before ambulatory measurements these steps should be demonstrated and practised with the patient beforehand in the clinic:

1. If applicable, attach the ECG electrodes.
2. Secure the device to the thorax using the elasticated thorax belt (with the **two** built in pockets). Apply the strap whilst in the standing position and exhale. A correctly fitted strap should not slip up or down once fastened. To record abdominal breathing, attach the abdominal strap around the stomach in the same way. Always ensure the straps are fitted over the night clothes and not directly onto bare skin.
3. Check the position of the pressure transducers (black rubber pads). The sensors should be fully inserted into the built-in pockets sewn onto the strap. Never pull on the tubing connected to the pressure pads.
4. Apply the flow prong and/or respiratory thermistor to the nose or fix the adapter hose to the oxygen mask.
5. If applicable, attach the neurologic electrodes (EEG, EOG, EMG, EXG), and the microphone.
6. Apply the finger sensor (SpO<sub>2</sub>) to a finger and secure the cable to the wrist. Whilst doing so, ensure that you do not apply pressure to the finger thereby disturbing the blood flow.
7. The device will automatically switch itself on at the predetermined time. Both LEDs light up briefly for testing purposes.  
If all sensors have been connected correctly, the red LED will go out. The green LED blinks at regular 4 second intervals.  
The red LED signals a faulty pulse signal. Check the finger sensor and cable.

Once the patient has been educated on fitting the device, the device can then be packed into its carry case for the patient to take home. The patient can refer to the quick reference guide when fitting the device at home.

#### **When going to bed the patient needs to:**

- Attach the device using the elasticated carrying strap.
- Attach the flow prong and finger sensor.
- If applicable, attach the abdominal strap and all other sensors and electrodes.
- Check that the sensors and their connectors are positioned correctly.

### **The next morning:**

- The patient has to detach the device and sensors and return all parts to the case. Cleaning of the sensors will be carried out by the technical staff at the clinic or hospital.
- Bring the case back to the clinic or hospital.
- For automatic evaluation the measurement can be transferred to the PC using the device software. Select the "Record" tab followed by "Read measurement from device".

### 3.5. Online Measurements (Inpatient)

Please adhere to the following steps when performing an online measurement:

1. Ensure the battery is fully charged (see Page 25).
2. Apply the device and connect all sensors to the patient (see Page 25)
3. Start the software on your PC.
4. Complete the Patient data fields in the menu item "Record / Start recording (Online)" and start the transfer.
5. Connect the device to the galvanic separation PC using the USB interface cable.
6. Perform bio signal calibration (see the online help of the software).
7. Once the test is complete, (the following morning) click the stop button  to end the measurement.
8. Detach the device from the USB connection

## 4. Service and Maintaining the device

### 4.1. Charging the battery

**Caution: It is not allowed to use a different battery charger than the one provided!  
The charger must not be operated in the patient's environment!**

#### 4.1.1. Charging Cradle

To charge the battery, the battery compartment on the back of the device can be opened and the battery removed. The battery charging adaptor included with the device is specially designed for this type of battery and should only be used with the device battery. It is recommended to charge the battery completely after each measurement.

#### Battery charging procedure:

- Remove the battery from the device and insert it into the battery cradle.
- Plug the charger into a power outlet
- The charger LED will illuminate orange. As soon as the battery is charged, the LED will illuminate green. The device can remain connected to the charging adaptor for a long period of time without suffering any damage.

#### 4.1.2. Charger

A power supply unit is optionally available for charging the battery. This is specially matched to the device's battery.

Battery charging procedure:

- Connect the battery charger to the device socket marked with the symbol 
- Plug the charger into a power outlet
- The integrated LED's in the top cover of the device displays the charging status of the batteries.

The device can remain connected to the charger adaptor for a long period of time without suffering any damage.

**4.1.3. Sonata Base**

Optional the battery of the device can be charged via the Sonata Base.

The following applies to charging the battery:

- During an online recording, each charging is stopped
- After an online recording, the charging by Sonata Base will take three hours to start
- After connecting the device to the Sonata Base, charging will start after three hours
- Charging the battery at the Sonata Base can be started or stopped at any time by pressing the battery button twice

The charging condition of the battery is displayed at the top of the device 

**Caution: When starting a measurement the charging process between the Sonata Base and the Sonata / Scala will be stopped!**

**4.2. Cleaning instructions****General**

As with any medical device, when using the Sonata / Scala system, certain hygienic procedures are required for safe reuse on the patient. Reusable products must have safe disinfectability to exclude any risk of infection from subsequent users / patients. The regulations of the Medical Devices Act stipulate that for such medical devices disinfection activities with procedures according to the RKI guideline are to be carried out in analogy to the methods of surface or instrument disinfection. Sterilization of the products listed below is not necessary.

The Sonata/Scala system is a medical device that is in the patient's environment when used as intended, with direct contact with the hands of the staff and the patient. These products are therefore classified as so-called "non-critical medical products" in accordance with the guidelines for hospital hygiene and infection prevention.

The cleaning and disinfection instruction for the device can be found in the enclosed "**Hygiene Plan Sonata / Scala**".

**Warning:**

No liquid should enter the device or plug connections. The sensors must not be immersed in liquid. Before cleaning, close the pressure connections of the device with caps!

Disconnect the charging and data cable from the device before each cleaning.

**In the washing machine**

The thorax and abdominal straps can be washed in the washing machine at 60° C (use a laundry net!). Let the straps dry in the air.

**Attention:**

- Do not use a dryer
- Remove thoracic and abdominal sensor beforehand

**Disposables**

Disposables must not be used multiple times or on different patients. Follow the guidelines for waste disposal in hospitals.

**4.3. Maintenance**

The device is maintenance-free. It is recommended to have a technical inspection carried out by the manufacturer or an agency authorized by the manufacturer every 24 months.

In order to maintain basic safety and essential performance, no special measures need to be taken with regard to EMC.

**4.4. Transport and storage conditions**

The ambient temperatures for transport and storage are between -25 °C and +70 °C, where:

- -25 °C and +5 °C without restrictions,
- + 5 °C to +35 °C with a relative humidity of up to 90%, without condensation,
- > 35° C to 70 °C at a water vapor pressure of up to 50 hPa.

Condensation must always be avoided.

## 5. Evaluation software for PC

### 5.1. Installing the software

To install the PC software, place the supplied CD into the CD-ROM drive or insert the supplied flash drive into the USB port of your computer. The program will automatically run. If the program does not run automatically open the Explorer option on your operating system and select the CD-ROM drive or flash drive. Once selected double click the Setup.exe file from the software.

You can check your computer and network (if applicable) for PC software versions that may have been previously installed. To ensure the correct up to date version is selected, open the software installed and click the Search button. The program will then automatically find and use the latest version installed.

Note: Alternatively you can update the software by selecting menu item Tools / Update from the software. (Only for updating the software).

After selecting the target drive the installation program will now automatically create the necessary directories and copy all the required files to the hard drive. The installation will automatically insert a new group containing an entry for the new software in the Programs folder of the Start Menu. In addition a shortcut on your Windows desktop will be created.

After successful installation, remove the CD or flash drive and store it in a safe place.

### 5.2. Selecting a printer

The measurement curves and reports can be printed on any commonly available type of printer (ink jet, dot-matrix or laser printer) that is already installed on your Windows system. The correct printer driver can be selected using the Windows Control Panel application.

## 6. Troubleshooting

**Channels (e.g. pressure) are missing in the display.**

*The channels are deactivated and therefore do not appear during "Test" and "Record".*

Check which channels are active for recording using the menu item Options / Channel settings.

**Channels are missing after loading a measurement.**

They were not recorded or have been hidden for measurement data display.

**Flow signal is missing during recording or follows the edge of the range.**

*The flow signal follows a straight flat line in the middle of the channel.*

Check the flow prong on the patient and the connection on the device. When measuring during PAP therapy, check the connection of the PAP adapter hose on the oxygen mask and on the device.

**Signal amplitudes are very small or do not appear.**

Check the corresponding sensors on the patient and their connections to the device. Whilst doing so, check that the tube hoses and the black pressure pads of the thorax sensors are intact. A leaking sensor can cause inaccurate readings in the thorax channel. The pressure pads should be attached to the patient securely without over tightening the belt and the tube hoses should be routed in such a way that they cannot kink.

Check all cables and connectors on the device and the PC.

**Pulse oximeters not responding.**

*The channels for oxygen saturation and pulse frequency are registering 0% and 0 P/min respectively despite the patient being connected.*

First check that the finger sensor is seated correctly on the patient and remove any nail polish that may be present. A small red light should be present in the sensor when the finger is inserted. If the light doesn't switch on when the sensor is applied to the finger, then you need to check the connection at the device and any intermediate extensions.

**Printout not working.**

*The printer prints characters on the page but in no apparent format.*

The wrong printer or printer driver has been installed.

*The printer is not responding to the print command.*

Check the printer cable and the connections to the printer and PC. The printer should be switched on and operational, i.e. the control LED's on the printer should be lit.

**Cannot establish connection to the device**

*The USB cable is not connected correctly.*

Check the connection of the USB cable to the device and the PC.

*USB interface was deactivated*

The USB interface can be activated in the PC software. Select the "Device connection" tab followed by menu "Settings".

*Battery is discharged.*

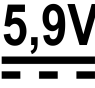
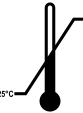
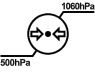
Charge the battery correctly.

**For clarification and in case of problems during the installation, maintenance or use, please refer to your local distribution partner, or directly to the manufacturer.**

## 7. Technical specifications

- Dimensions
  - Patient recorder : 30mm x 77mm x 135mm (H x W x L)
  - Neuro module : 30mm x 62mm x 70mm (H x W x L)
- Weight
  - Patient recorder : 230g including storage battery
  - Neuro module : 80g
- Housing : plastic (ABS, UL 94HB)
- Temperature range : +5°C...+40°C (operation)
- Moisture : 10% - 90% (operation)
- Atmospheric pressure : 70kPa - 106kPa (operation)
- Storage media : Internal SD card
- Storage capacity : min. 100 hours / 1kHz
  
- Registered parameters:
  - Respiratory activity : Differential pressure measurement via flow prong (with adaptor also during PAP therapy), alternative or additional measurement by means of thermistor possible
  - Thoracic effort : Differential pressure measurement by means of rubber cuffs built into chest strap; alternative measurement by RIP belt possible
  - Abdominal effort : Differential pressure measurement by means of rubber cuffs built into abdomen strap; alternative measurement by RIP belt possible
  - Breathing sounds : Phonometric transducer via flow prong  
Alternative or additional measurement with external microphone possible
  - SpO<sub>2</sub>/Pulse : Built-in pulse oximeter, calibrated for functional oxygen saturation  
SpO<sub>2</sub> measurement range: 80%-100% ± 2% SpO<sub>2</sub> 60%-79% ± 4% SpO<sub>2</sub>  
Pulse measurement range: 50 1/min - 150 1/min ± 2%  
(Reference electrical pulse simulator)  
Finger sensor special rubber-coated thimble finger sensor
  - Pulse wave : Plethysmogram display; measurement via finger sensor
  - Position : Acceleration sensor for position recording (5 positions)
  - Ambient Light : Photometric measurement and light-Intensity display
  - PAP : Differential pressure measurement directly on oxygen mask  
Measurement range: 0cmH<sub>2</sub>O - 45cmH<sub>2</sub>O ± 5%
  - Leg movement : Two bipolar leg channels (EMG) for measurements of muscular actions, connection via 1.5mm safety plug (only Sonata)
  - ECG : 6 channel lead via adhesive electrodes (only Sonata), also suitable for patients < 10kg
  - Central heart frequ : Measurement range: 30 1/min - 200 1/min ± 2% (only Sonata)
  - PTT : Measurement range: 100 ms – 355 ms ± 4% (only Sonata)
  - Syst. blood pressure : Trend (only Sonata)
  - EEG : 6 channel lead via adhesive electrodes; connection via 1.5mm safety plug;  
Impedance: 10 MΩ, frequency: 0,2 Hz - 45 Hz (only Sonata)
  - EOG : 2 channel lead via adhesive electrodes for left and right eye; connection via 1.5mm safety plug (only Sonata)
  - EMG : Bipolar measurements (3 x EMG) of muscular actions of chin via adhesive electrodes
  - ExG : 2 bipolar or 4 unipolar neurologic channels, connection via 1.5mm safety plug
  - Actimeter : 2 actimeter to record patient movement activity (only Sonata)
- Fault indicator : LED on top of the device
- Operation frequency : 2400 - 2472 MHz
- Modulation : DSSS (BPSK, QPSK, CCK), OFDM (BPSK, QPSK, 16QAM, 64QAM)
- RF output power : 13 dBm max. (2400-2483.5 MHz) IEEE802.11 b/g/n mode
- Power supply : Rechargeable Li-Ion storage battery 3.8 V
- Output : USB interface with cable for data transmission
- Power consumption : Max. 260mA
- Online operations : In online operation with a patient, a galvanic isolation to the PC is indispensable.

## 8. Used Symbols

<i>Symbol</i>	<i>Meaning</i>	<i>Symbol</i>	<i>Meaning</i>
	Pay attention to the instructions for use!		Electrical and electronic devices may not be disposed of with domestic waste. Consumers are obliged by law to return electrical and electronic devices at the end of their service lives to the public collection points set up for this purpose or point of sale. Details to this are defined by the national law of the respective country. This symbol on the product, the instruction manual or the package indicates that a product is subject to these regulations. By recycling, reusing the materials or other forms of utilising old devices, you are making an important contribution to protecting our environment.
	Medizinprodukt		
	Type BF		
	Protection class II		
	Best before date		
	Manufacturer	<b>IP22</b>	Device is protected against solid foreign bodies with diameter $\geq 12.5$ mm. Device is protected against access with a finger. The device is protected against falling dripping water when the housing is tilted up to $15^\circ$ .
	Manufacturing date		Once product! Neither suitable for a preparation nor for a multiple application.
	Catalog number		Serial number
	Unique Device Identifier	<b>5,9V</b> 	Maximum permitted charging voltage
	Temperature limit		Air pressure limitation

## 9. EMC Information

**Warning:** Medical electrical devices subjected to special preventive measures regarding to electromagnetic compatibility (EMC). They must be put in operation and installed according to the EMC information in this manual. Portable and mobile RF communications equipment (e.g. mobile phones) can influence medical electrical devices. Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result. For an intended use of the device, the specified equipment in this manual must be used. Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

<b>Guidelines and Manufacturer's Declaration – Electromagnetic Emissions</b>		
The device is intended for use in an electromagnetic environment as specified below. The customer or user of the device should ensure that it is used in such an environment.		
<b>Emission Test</b>	<b>Compliance</b>	<b>Electromagnetic Environment - Guidelines</b>
RF – Emission according to CISPR 11 30 MHz - 1 GHz	Group 1	The device uses RF energy only for it's internal function. So it's RF emission is very low and it's implausible that it cause any interferences to surrounding electronic devices.
RF – Emission according to CISPR 11 30 MHz - 1 GHz	Class B	The device is suitable for the use in all establishments, including domestic establishments and those, which are connected to the public power supply, which powers buildings for residential purposes.
Harmonic Emissions according to IEC 61000-3-2	Not applicable	
Voltage Fluctuations / Flicker Emissions according to IEC 61000-3-3	Not applicable	

Table 1: Table 201 EN 60601-1-2, Electromagnetic Emissions

<b>Guidelines and Manufacturer's Declaration – Electromagnetic Immunity</b>			
The device is intended for the electromagnetic environment as specified below. The customer and user of the device should ensure that it is used in such an environment.			
<b>Immunity test</b>	<b>Test Specification</b>	<b>Compliance level</b>	<b>Electromagnetic Environment - Guidance</b>
Electrostatic Discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 15 kV air	± 8kV contact ± 2kV, ±4kV, ±8kV, ±15kV air	Floors should consists of wood, concrete or should be covered with ceramic tiles. If the floor is covered with synthetic material, the relative humidity must be at least 30 %.
Electrical fast transient / burst IEC 61000-4-4	± 1kV for input / output lines 100 kHz repetition frequency  ± 2kV for power supply lines 100 kHz repetition frequency	± 1kV  ± 2kV	
Power Frequency (50/60 Hz) Magnetic Field IEC 61000-4-8	30 A/m	30 A/m	

Table 2: Table 4, 5, 7, 8 EN 60601-1-2, Electromagnetic Immunity

**Guidelines and Manufacturer's Declaration – Electromagnetic Immunity**

The device is intended for an electromagnetic environment as specified below. The user of the device should make sure that is used in such an environment.

Immunity test	Test Specification	Compliance level	Electromagnetic Environment - Guidance
Conducted RF disturbance IEC 61000-4-6	3 V <sub>eff</sub> 150kHz - 80MHz 6 V <sub>eff</sub> ISM/Amateur bands 80 % AM / 1 kHz	3 Vrms outside the ISM band, 6 Vrms in the ISM and amateur radio bands  3 Vrms outside the ISM band, 6 Vrms in the ISM band	<p><b>Warning:</b> Interferences may occur in the environment of devices, which are marked with the following symbol:</p> 
Radiated RF disturbance IEC 61000-4-3	10 V/m 80MHz - 2,7 GHz 80 % AM / 1 kHz	10 V/m 80 MHz to 2,7 GHz	
Proximity fields from RF wireless communications equipment IEC 61000-4-3	385 MHz (18 Hz Pulse Modulation (PM))	27 V/m	
	450 MHz (18 Hz PM)	28 V/m	
	710 MHz (217 Hz PM)	9 V/m	
	745 MHz (217 Hz PM)	9 V/m	
	780 MHz (217 Hz PM)	9 V/m	
	810 MHz (18 Hz PM)	28 V/m	
	870 MHz (18 Hz PM)	28 V/m	
	930 MHz (18 Hz PM)	28 V/m	
	1720 MHz (217 Hz PM)	28 V/m	
	1845 MHz (217 Hz PM)	28 V/m	
	1970 MHz (217 Hz PM)	28 V/m	
	2450 MHz (217 Hz PM)	28 V/m	
	5240 MHz (217 Hz PM)	9 V/m	
	5500 MHz (217 Hz PM)	9 V/m	
	5785 MHz (217 Hz PM)	9 V/m	

Table 3: Table 4, 5, 7, 8 EN 60601-1-2, Electromagnetic Immunity

Guidelines and Manufacturer's Declaration – Electromagnetic Immunity						
Test frequency MHz	Frequency band <sup>a</sup> MHz	Radio service <sup>a</sup>	Modulation <sup>b</sup>	Max. Performance W	Distance m	Immunity test level V/m
385	380 - 390	TETRA 400	Pulse Modulation <sup>b</sup> 18 Hz	1,8	0,3	27
450	430 - 470	GMRS 460, FRS 460	FM <sup>c</sup> ± 5 kHz Hub 1 kHz Sinus	2	0,3	28
710 745 780	704 - 787	LTE Band 13, 17	Pulse Modulation <sup>b</sup> 27 Hz	0,2	0,3	9
810 870 930	800 - 960	GSM 800/900 TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse Modulation <sup>b</sup> 18 Hz	2	0,3	28
1 720 1 845 1 970	1 700 - 1 990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse Modulation <sup>b</sup> 217 Hz	2	0,3	28
2 450	2 400 - 2 570	Bluetooth, WLAN 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse Modulation <sup>b</sup> 217 Hz	2	0,3	28
5 240 5 500 5 785	5 100 - 5 800	WLAN 802.11 a/n	Pulse Modulation <sup>b</sup> 217 Hz	0,2	0,3	9

NOTE: If necessary, to achieve immunity test levels, the distance between the transmit antenna and the device can be reduced to 1 m. The 1 m test distance is permitted according to IEC 61000-4-3.

a For some radio services, only the frequency for the radio link from the mobile communication device to the Sonata Base (en: uplink) has been included in the table. The carrier must be modulated with a square wave signal with 50% duty cycle.

b The carrier must be modulated with a square wave signal with 50% duty cycle.

c As an alternative to Frequency Modulation (FM), pulse modulation at 50% duty cycle at 18 Hz may be used, as this, if not the actual modulation, would be the worst case scenario

Table 4: Table 9 EN 60601-1-2, Test specifications for the immunity of enclosures to high-frequency wireless communication devices

**10. Index**

Abdominal sensor.....	23
Accessories.....	20, 22
Assigned Purpose.....	20
Attaching the device.....	25
Attaching the sensors.....	24
Battery charging.....	24
Charging the battery.....	25
Cleaning instructions.....	26
Disposal.....	22
EEG.....	24
EMG.....	24
EOG.....	24
Equipment.....	20
EXG.....	24
Finger sensor.....	23
Flow prong.....	23
Flow sensor.....	23
Functional testing.....	22
Information.....	21
Installing the software.....	27
Leg movement.....	24
Maintenance.....	26
Operating elements.....	24
Optical isolator.....	21
PAP.....	20
PAP sensor.....	23
PDA.....	23
Position sensor.....	23
Preparing for measurement.....	24
Pressure Diff Adapter.....	23
Pressure pads.....	23
Safety-related information.....	21
Selecting a printer.....	27
Sensors.....	23
Starting the measurement.....	25
System requirements.....	20
Technical inspection.....	22
Technical specifications.....	28
Thermistor.....	23
Third-party accessories.....	22
Thorax sensor.....	23
Transport and storage conditions.....	26
Troubleshooting.....	27
Warranty.....	22
Whole system.....	20

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## Ordering information

Art.-No.	Spare parts / consumer material / Accessories
930300	<b>Finger sensor premium SpO2</b> finger cuff with cable for long time measurement, M 9/10
930320	<b>Thermistor</b> , with fixation for flow prong, reusable, M10
930321	<b>Snoring microphone</b> , external, reusable, M10
930322	<b>Pressure Diff Adapter</b>
930253	<b>Shoulder bag</b> cloth bag with shoulder strap for carrying device, M 10
930393	<b>Effort sensor, close to device</b> , 2 pressure pads with flexible connection tubes, M 10
930394	<b>Effort sensor, away from device</b> , 2 pressure pads with flexible connection tubes, M 10
930397	<b>Effort sensor, close to device, 2m</b> , 2 pressure pads with flexible connection tubes, for a position of the device behind the patient, M 10
930398	<b>Effort sensor, away from device, 2m</b> , 2 pressure pads with flexible connection tubes, for a position of the device behind the patient, M 10
930260	<b>Carrying strap</b> , flexible, blue, <b>Size S</b> , M 10
930261	<b>Carrying strap</b> , flexible, black, <b>Size M</b> , M 10
930262	<b>Carrying strap</b> , flexible, red, <b>Size L</b> , M 10
930263	<b>Carrying strap</b> , flexible, blue, with connection to position sensor 930310, <b>Size S</b> , M 10
930264	<b>Carrying strap</b> , flexible, black, with connection to position sensor 930310, <b>Size M</b> , M 10
930265	<b>Carrying strap</b> , flexible, red, with connection to position sensor 930310, <b>Size L</b> , M 10
927370	<b>RIP Belt</b> , flexible, black, reusable
930324	<b>RIP-Interface cable Thorax / Abdomen</b> , with Key hole plug, 1m, M 10
930330	<b>Actimeter</b> , M 10
930310	<b>External position sensor</b> , M 10
930200	<b>Sonata Base</b>
930220	<b>Intercom-Cable</b> Connection Cable device - Neuro module, 8.5 cm
930221	<b>Intercom-Cable</b> Connection Cable device - Neuro module, 60cm
930224	<b>USB Cable</b> for data transmission to PC, 1.8m, M 9/10
930230	<b>Electrical isolator USB</b> for galvanic separation between device and PC during online measurement; complete with adapter, M 9/10
930226	<b>USB Cable</b> for data transmission between device and Sonata Base, 3m, M 10
930236	<b>Adapter cable Redel-Hirose</b> , for data transmission between device and Sonata Base, 25 cm, M 10
930237	<b>USB cable Redel - Redel</b> , for data transmission between device and Sonata Base, 2.7 m, M 10
930238	<b>USB cable Redel - Redel</b> , for data transmission between device and Sonata Base, 3.6 m, M 10

Art.-No.	Spare parts / consumer material / Accessories
930250	<b>Carrying console</b> for patient recorder, M 10
930251	<b>Carrying console</b> for Neuro module
930252	<b>Carrying console V-form</b> for patient recorder and Neuro module
930180	<b>Li-Ion battery Patient Recorder</b> , M 10
930204	<b>Charging cradle</b> for battery, M10
930209	<b>Power supply, medical, for Patient Recorder</b> , M 9/10
930375	<b>Cleaning set PSG</b> , M10
930376	<b>Cleaning set PG</b> , M10
930346	<b>Instruction for Use Porti 10, GE-EN-FR-IT</b>
930391	<b>Instruction for Use Sonata/Scala, GE-EN-FR-IT</b>
930349	<b>Instruction for Use VitalNight PSG, GE-EN-FR-IT</b>
Art.-No.	Consumer material
200-0312 /10	<b>Flow prong</b> nose spectacles for flow measurement
925235	<b>CPAP Adapter incl. PVC tube, Luer-Lock, 80 cm</b> , disposable
925236	<b>CPAP Adapter incl. PVC tube, Luer-Lock, 200 cm</b> , disposable
Art.-No.	Spare Parts / Accessories for use with children
930395	<b>Effort sensor for children, complete</b> , 2 pressure pads with flexible connection tubes for positioning the device behind the child, M 10
930266	<b>Carrying strap for children</b> , flexible, black, with connection to position sensor 930310, <b>Size XS</b> , M 10
930267	<b>Carrying strap for children</b> , flexible, black, with connection to position sensor 930310, <b>Size XXS</b> , M 10
930268	<b>Carrying strap for children</b> , flexible, black, with connection to position sensor 930310, <b>Size XXXS</b> , M 10
920126	<b>Infant sensor SpO2</b> with Philips connector (has to be extended with Hirose- Philips extension 920122), infants 1 - 4 kg
920127	<b>Infant sensor SpO2</b> with DSUB connector (has to be extended with Hirose-DSUB extension 920123), infants 4 - 40 kg
920122	<b>Extension Hirose - Philips</b> for device and infant sensor
920123	<b>Extension Hirose - DSUB</b> for device and infant sensor

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**Benannte Stelle** : **mdc medical device certification GmbH**