

# Cleaning instructions

How Should I Clean my Mask?



## Nasal masks

Useful life up to 12 months\*



CARA



JULIA



JOYCEone



JOYCEeasy

Material of mask parts	Wash by hand in warm water with mild detergent**	Dishwasher up to 70 °C
Plastic	daily	weekly
Silicone	daily	weekly
Textile	weekly	-

## Full face masks

Useful life up to 12 months\*



CARA Full Face



LENA



JOYCEone Full Face



JOYCEeasy Full Face



JOYCEeasy next Full Face

Material of mask parts	Wash by hand in warm water with mild detergent**	Dishwasher up to 70 °C
Plastic	daily	weekly
Elbow with anti-asphyxia valve	daily	-
Silicone	daily	weekly
Textile	weekly	-

## Silentflow 3

Useful life 6 to 12 months\*

Components	Wash by hand in warm water with mild detergent**	Dishwasher up to 70 °C
Connecting sleeve	weekly	weekly
Center piece	weekly	weekly

### Basic instructions:

These abridged instructions do not replace the user manuals for individual products.

\* Useful life depends on cleaning and on the detergents used, on the amount of time worn daily, on therapy pressure, and on individual secretion of sweat. Using the second mask cushion (included as an option) does not extend useful life.

\*\* We recommend using a special cleaner, such as Löwosan, for daily cleaning.



You can find informative instructions, application examples and helpful tips on our YouTube channel:  
<https://www.youtube.com/@LoewensteinMedicalGlobal>

CE 0197



  
 Löwenstein Medical Technology  
 Kronsaalsweg 40  
 22525 Hamburg, Germany  
 T: +49 40 54702-0  
 F: +49 40 54702-461  
[info@loewensteinmedical.de](mailto:info@loewensteinmedical.de)

**Sales + Service**  
 Löwenstein Medical  
 Arzbacher Straße 80  
 56130 Bad Ems, Germany  
 T. +49 2603 9600-0  
 F. +49 2603 9600-50  
[info@loewensteinmedical.com](mailto:info@loewensteinmedical.com)  
[loewensteinmedical.com](http://loewensteinmedical.com)

